

Mirage Activa™ LT

NASAL MASK

Fitting Guide



- Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



- Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.
- Reconnect the lower headgear clip ensuring it clicks into place.



- Allow room for the cushion's 'active cell' to inflate. Use two fingers as a guide.




- Connect one end of the air tubing to the swivel and the other end to the airflow device.




- Turn on the airflow device.



- While lying down, adjust the dial for comfort and seal.

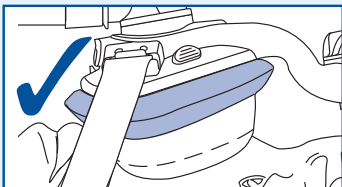

- If the mask is fitted correctly, the cushion should be smooth and the active cell (highlighted) fully inflated.

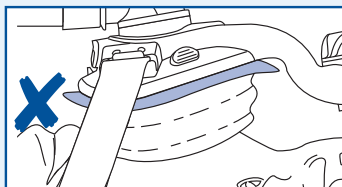

- If fitted incorrectly, the cushion will be creased and the active cell underinflated.

⚠ For details, refer to the *Mirage Activa LT Nasal Mask User Guide*.