

Mirage Micro™

NASAL MASK

Fitting Guide



- Turn the MicroFit dial to position 24, where the forehead support is fully extended.



- Disconnect one headgear clip, place the mask on the face and pull the headgear over the head.



- Bring the lower straps below the ears.
- Connect the lower headgear clip ensuring it clicks into place.



- Unfasten the upper Velcro™ tabs, pull the headgear straps evenly until comfortable, then reattach the Velcro.
- Do not overtighten the straps.



- Repeat Step 4 with the lower headgear straps.
- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.



- While lying down, turn the MicroFit dial until the mask is evenly and comfortably positioned on the face.
- If any air leaks are felt, readjust the dial and/or headgear.

⚠ For details, refer to the *Mirage Micro Nasal Mask User Guide*.

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** ResMed Corp 14040 Danielson Street Poway CA 92064-6857 USA, ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide. Protected by patents: AU 710733, AU 741003, AU 766623, AU 775051, AU 777033, AU 785376, CA 2261790, EP 0956069, EP 1187647, EP 1187648, EP 1187649, EP 1187650, EP 1314445, EP 1479406, EP 1582230, EP 1640034, HK 1057714, JP 3686609, NZ 513052, NZ 526165, NZ 526166, NZ 526167, NZ 526168, US 6112746, US 6119693, US 6357441, US 6374826, US 6439230, US 6463931, US 6532961, US 6557556, US 6581594, US 6581602, US 6634358, US 6691708, US 6823865, US 6860269, US 6871649, US 6997188, US 7011090, US 7159587, US 7178527, US 7188620, US 7216647, US 7234466, US 7243651. Other patents pending. Protected by design registrations: AU 304309, AU 319212, AU 319213, AU 319214, AU 319215, AU 319266, AU 319267, AU 319216, AU 319217, AU 319185, AU 319218, AU 320813, AU 320814, CN200530117106.1, EP366844, JP1264474, NZ406357, USD532512, USD547439, USD545960. Others pending. Mirage and Mirage Micro are trademarks of ResMed Ltd and Mirage is registered in U.S. Patent and Trademark Office. Velcro is a trademark of Velcro Industries B.V. Specifications may change without notice. © 2008 ResMed Ltd. 1013266/1 08 10