

# Welcome

Thank you for choosing the VPAP Adapt or H5i. Before operating this device, please read the entire Welcome and Information Guides.



## VPAP Adapt at a glance

The VPAP Adapt system comprises the following elements:


- VPAP Adapt device
- Air tubing
- 90W power supply unit
- S9Travel bag
- SD card
- S9 SD card protective folder.

Optional components include:

- H5i heated humidifier
- Standard air tubing
- SlimLine air tubing
- 3m air tubing
- ClimateLine heated air tubing
- ClimateLine<sup>MAX</sup> heated air tubing
- 30W power supply unit (does not support H5i)
- Power Station II battery pack
- DC/DC Converter 24V/90W.

## Traveling with the VPAP Adapt

When traveling you may choose to leave behind your H5i and take your VPAP Adapt device only. If this is the case, ensure you pack the SlimLine, Standard or 3m air tubing in your travel bag, as the ClimateLine or ClimateLine<sup>MAX</sup> heated air tubing is not designed to connect directly to the VPAP Adapt device.

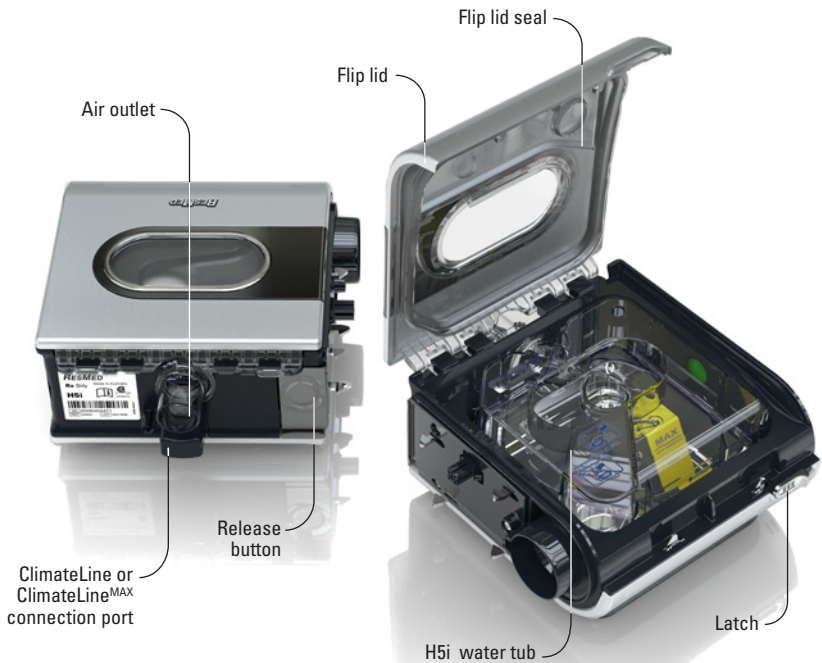

## H5i at a glance

The H5i system comprises the following elements:

- H5i heated humidifier
- H5i water tub
- ClimateLine<sup>MAX</sup> heated air tubing (if sold as a Climate Control Kit)


Optional components include:

- ClimateLine heated air tubing
- H5i cleanable water tub.

## Traveling tips

When moving or traveling with your H5i:

- Ensure that the water tub is empty.
- Disconnect the H5i from the VPAP Adapt by using the release button.

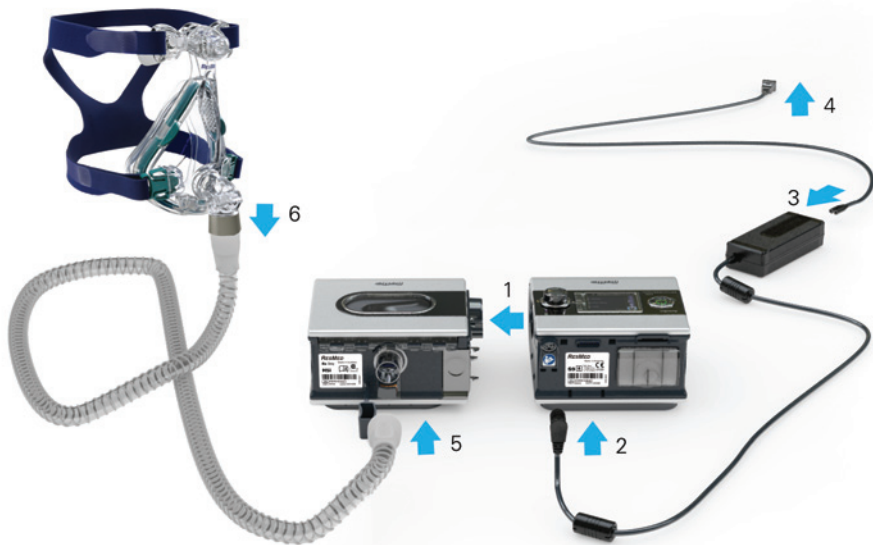

## Setup


1. Align your H5i with your VPAP Adapt and push them together until they click into place.
2. Connect the DC plug of the power supply unit to the rear of the VPAP Adapt.
3. Connect the power cord to the power supply unit.
4. Plug the other end of the power cord into the power outlet.
5. Connect one end of the air tubing firmly onto the air outlet.
6. Connect the assembled mask system to the free end of the air tubing.

### Notes:

- For more information on assembling your mask see your mask user guide.
- Recommended masks are available on [www.resmed.com](http://www.resmed.com) on the Products page under Service & Support.

## Filling the water tub

1. Slide the latch and lift open the flip lid.
2. Remove the water tub.
3. Fill the water tub (through the center hole) with distilled or deionized water up to the maximum water level mark (380 mL).
4. Return the water tub to the H5i.
5. Close the flip lid ensuring that it clicks into place.

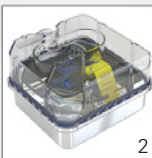



**LCD screen**  
Displays the menus,  
treatment screens and  
reminders.

**Start/ Stop  
button**  
Starts or stops  
treatment.

**Info menu**  
Allows you to  
view your sleep  
statistics or exit  
from the menu.

**Setup menu**  
Allows you to  
make changes  
to settings or  
exit from the  
menu.

**Push dial**  
Turning the dial  
allows you to  
scroll through the  
menu and change  
settings. Pushing the  
dial allows you to  
enter into a menu and  
confirm your choice.

**Key:**

Home

Humidity level

Ramp

Heated tube

Climate Control

Start/ Stop

Push dial

Setup menu

Info menu


## VPAP Adapt basics

The control panel of your VPAP Adapt device includes an LCD screen as well as a Start/Stop button, Push dial and the Info and Setup menu keys.


## Adjusting humidity level

Ranging from OFF to 6, you can adjust the humidity level at any time to find the setting that is most comfortable for you. To adjust your level of humidity:

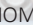


1. From your HOME screen, turn  until the HUMIDITY LEVEL screen is displayed in blue.




2. Press . The selection is highlighted in orange.



3. Turn  until the humidity level that you require is displayed.






4. Press  to confirm your choice.

## Warming the H5i

The warm-up feature is used to pre-heat the water before starting treatment.

To start warming:

1. Navigate to the humidity level icon.
2. Push and hold  for three seconds. The Warming Up status bar is displayed.

You can stop warm-up at any time by pressing  for at least three seconds. You can start treatment at any time by pressing .


**Note:** In order to assist the heater plate in cooling, your VPAP Adapt device will continue to blow air for up to an hour after treatment has stopped. However, you can unplug the device from the power outlet at any time and allow the heater plate to cool without air flow.




## Adjusting ramp time

Designed to make the beginning of treatment more comfortable, ramp time is the period during which the pressure increases from a low start pressure to the treatment pressure. To adjust your ramp time:




1. From your HOME screen, turn  until the RAMP screen is displayed in blue.



2. Press . The selection is highlighted in orange.




3. Turn  until the ramp time you require is displayed.



4. Press  to confirm your choice.


## Using mask-fit

You can use mask-fit to help you fit your mask properly. This feature delivers treatment pressure for a three-minute period, prior to starting treatment, during which you can check and adjust your mask-fit to minimize leaks.

1. Fit your mask as described in the mask user guide.
2. From your HOME screen, push and hold  for three seconds.

One of the following MASK FIT screens is displayed:



3. If necessary, adjust your mask, mask cushion and headgear until you have a Good mask-fit. After three minutes, the pressure reverts to the set pressure and treatment will begin. You can end mask-fit at any time by pressing .


## Selecting the mask type

To select your type of mask:




1. Press . The SETTINGS screen is displayed.




2. Turn  until Mask is displayed in blue.




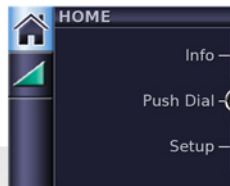
3. Press . The selection is highlighted in orange.





4. Turn  until the mask setting that you require is displayed.





5. Press  to confirm your choice.



6. Push  to return to the HOME screen.



## Getting started

1. Make sure the power is connected.
2. Adjust the ramp time or humidification level if required.
3. Fit your mask as described in your mask user guide.
4. To start therapy, simply breathe into the mask and/or press .
5. Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.
6. To stop treatment at any time, remove your mask and/or press .

### Notes:

- *If your clinician has enabled SmartStart your device will start automatically when you breathe into the mask and stop automatically when you remove your mask.*
- *If power is interrupted during treatment, the device automatically restarts therapy when power is restored.*

## Viewing the treatment screens

Depending on how the system has been configured, you will see one of the following example screens once therapy has started:




- ✓ H5i humidifier



- ✓ H5i humidifier
- ✓ ClimateLine or ClimateLine<sup>MAX</sup> heated air tube
- ✓ Climate Control – Auto



- ✓ H5i humidifier
- ✓ ClimateLine or ClimateLine<sup>MAX</sup> heated air tube
- ✓ Climate Control – Manual

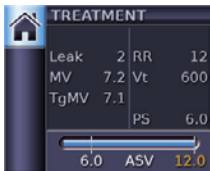

- ✓ Standard VPAP Adapt without optional accessories




- ✓ Oximetry data via the oximeter adapter

In ASV mode, the fixed lines on the pressure bar indicate the expiratory and inspiratory pressures. In CPAP mode, only the set pressure is shown.

## Viewing the Info menu

The Info menu has a series of screens showing your sleep quality, sleep report and service information.

To view the Info menu, push  from your HOME screen.

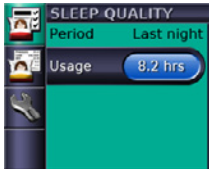

- ✓ In the Sleep Quality screen, you can always view your usage hours during the last session and if enabled by your clinician, data on mask-fit and AHI is displayed.
- ✓ In the Sleep Report, only the period can be changed—other values are for display only.
- ✓ In the Service information, the total device run hours (includes warm-up/cool-down times for the humidifier) and the software identifications are displayed.

To return to the HOME screen, push .


## Cleaning and maintenance

You should regularly carry out cleaning and maintenance as described in this section.

### Daily

1. Remove the air tubing by pulling on the finger grips on the cuff. Hang the air tubing in a clean, dry place until next use.
2. Slide the H5i latch.
3. Lift open the flip lid and remove the water tub.
4. Wash the H5i water tub in warm water using a mild detergent.
5. Rinse thoroughly in clean water and allow it to dry away from direct sunlight.
6. Before next use, fill the water tub, return it to the H5i and close the flip lid.

**Note:** Do not open the H5i water tub. There are no user serviceable parts inside.

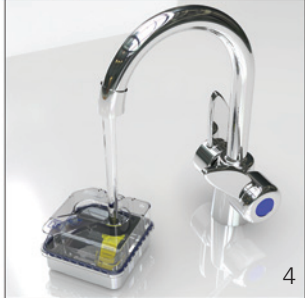

### Weekly

1. Remove the air tubing from the VPAP Adapt device and the mask.
2. Wash the air tubing in warm water using mild detergent.
3. Rinse thoroughly, hang, and allow to dry.
4. Before next use, reconnect the air tubing to the air outlet and mask.

### Monthly

1. Wipe the exterior of the VPAP Adapt and H5i with a damp cloth and mild detergent.
2. Check the air filter for holes and blockage by dirt or dust.
3. Peel the flip lid seal from the flip lid and wash it in warm water using a mild detergent.

## Maintenance checklist

- ✓ Inspect the H5i water tub and flip lid seal for wear and deterioration.
- ✓ Replace the water tub if any component is leaking or has become cracked, cloudy or pitted.
- ✓ Replace the flip lid seal if cracked or torn.
- ✓ Clean white powder deposits in the water tub by using a solution of one part household vinegar to 10 parts water.

## Replacing the air filter

Replace the air filter every six months (or more often if necessary).

1. Remove the air filter cover from the back of the VPAP Adapt device.
2. Remove and discard the old air filter.
3. Insert a new ResMed air filter ensuring that it is sitting flat in the air filter cover.
4. Replace the air filter cover.




## SD card

An SD card has been supplied to gather therapy data from your VPAP Adapt device and provide settings updates from your clinician. When instructed to do so, disconnect the VPAP Adapt device from the power outlet, remove your SD card, insert it into the protective folder and send it to your clinician.

### Removing the card

1. Push in the SD card to release it.
2. Remove the card.
3. Insert the card into the protective folder.
4. Send the protective folder back to your clinician as instructed.

For more information on removing and inserting your card refer to your S9 SD Card Protective Folder provided with your device. Please retain the S9 SD Card Protective Folder for future use.

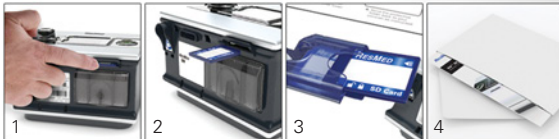

### Notes:

- For Troubleshooting information refer to the Information Guide.
- For more product information see [www.resmed.com](http://www.resmed.com).