RESMED

Gebrauchsanweisung
Instruction Manual
Mode d'emploi
Gebruiksaanwijzing
Istruzioni per l'uso
Εγχειρίδιο οδηγιών χρήσης
Kullanım Kılavuzu

Silent Papillon®

MAP
SERIES

Silent Papillon®

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This Instruction Manual applies for the Silent Papillon® nasal mask, the AeroFix® II headgear and the separately available AeroFix® plus tubing clip from MAP Medizin-Technologie GmbH. The figures are found inside this wrapper.

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Note

The instructions in this manual may differ from the product. This does not affect the product's performance. This instruction manual is protected by copyright. All rights reserved. Any reproduction including excerpts requires the express consent of MAP Medizin-Technologie GmbH.

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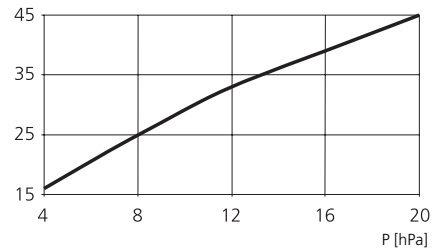
GB

Intended Use

The Silent Papillon® nasal mask is designed for the treatment of patients suffering from restrictive, obstructive, or central respiratory disorders who require nasal positive pressure therapy (nCPAP or biLEVEL). Examples of these conditions include sleep apnea, chronic restrictive or obstructive lung diseases, and rib cage deformations.

restrictive = limiting
obstructive = blocking
central = brain-triggered

The Silent Papillon nasal mask is a high-quality MAP Medizin-Technologie GmbH product, and has been developed on the basis of leading-edge research into respiratory therapies. Silent

Papillon is suitable for reuse by multiple patients following proper cleaning. (See Chapter "Cleaning, Disinfection and Sterilisation in Clinical Practice").

Before Using for the First Time

Please examine the figures on the inside front cover and read carefully all the information provided in the Instruction Manual. Hygienic use of the Silent Papillon nasal mask and product durability in accordance with the useful life specifications can be achieved only by strict compliance with these instructions.

Do not use the Silent Papillon nasal mask if the packaging is damaged. Check that the complete package contents are present. The package contents comprise the nasal mask, the AeroFix® II headgear [18], the clip fastener [17], and this Instruction Manual.

Clean all components and the headgear of the new mask before using them for the first time. See the "Disassembly of the Nasal Mask" and "Cleaning in the Home Environment" sections for more information. Wash the headgear separately for the first few times, as the color may run.

Legend

See Figures on inside front cover

- 1 Forehead pad, mint-green color
- 2 Mounting holes for forehead pad
- 3 Forehead frame
- 4 Frame base
- 5 Hinge grip pressure points
- 6 Mask frame
- 7 Exhalation system
- 8 Exhalation system air junction
- 9 Connector cover
- 10 External pressure measurement and oxygen connector
- 11 Air junction openings
- 12 Mask cushion, mint-green color
- 13 Tongue joint
- 14 Groove joint
- 15 Exhalation system insert tube
- 16 Slots to attach the headgear
- 17 Black clip fastener
- 18 AeroFix® II headgear
- 19 AeroFix® plus tubing clip
(not included in package contents)
- 20 Air tubing
- 21 Tubing connector
- 22 Scale of the four hinge positions

Safety Information

Please follow the instructions below to avoid possible risks to health and to ensure your respiratory therapy is fully effective.

- The Silent Papillon nasal mask may be used only for the delivery of nasal positive pressure therapy (nCPAP or biLEVEL).
- The nasal mask is suitable for delivering therapeutic pressures ranging between 4 and 20 hPa.
- The type of mask used affects the therapeutic pressure. If you have been treated with a different mask in the past, the treating physician should check whether the therapeutic pressure needs to be adjusted.
- Please contact your treating physician if you experience earache or pain in the nose or sinuses after starting your therapy, or if you have an allergic reaction to one of the materials used.
- Inhaled air is richer in oxygen (O₂) than CO₂-enriched exhaled air. To prevent exhaled air being rebreathed, this air must always be able to escape through the air junction [8] of the exhalation system [7] [see Figure C].

Before each use, ensure that the openings in the air junction are not blocked. Any deposits must be removed. For the exhalation system to function properly, it is essential that the mask cushion is assembled correctly. To ensure this, push the round opening of the mask cushion [12] over the air junction up to the limit stop [see "Assembly" section].

- The exhalation system operates only with the use of the insert tube, since otherwise too much air would escape, thus reducing the efficacy of the therapy.
- Do not use or repair a defective mask. Use only original spare parts manufactured by MAP Medizin-Technologie GmbH.
- When using replacement parts, please note the color of the mask cushion: use only mint-green mask cushions with the Silent Papillon model. The mint-green mask cushion for the Silent Papillon nasal mask may under no circumstances be replaced by the arctic-blue mask cushion with integral exhalation vents for the Papillon nasal mask.
- The Silent Papillon nasal mask exhalation system may under no circumstances be exchanged for the Papillon nasal mask connection system.
- The mask should be taken off as soon as you switch off your respiratory therapy device, or alternatively the mask and air tubing should be disconnected. This will prevent you from rebreathing exhaled air.
- External pressure measurement is possible only if the external pressure measurement connector [10] is clean and dry.
- If the external pressure measurement connector is not in use, it must always be sealed by the connector cover [9].
- Avoid excess formation of condensation, particularly at low therapeutic pressures. If necessary, reduce the settings of the air humidifier accordingly.
- To remove the mask safely, first open the clip fastener [17] on the headgear [18] and separate the tubing connector [21] from the exhalation system air junction [8]. Do not pull on the mask.
- Follow the instructions on regular cleaning of the mask to prevent possible respiratory tract infections.

Disassembly of the Nasal Mask

Detach the forehead frame [3] from the frame base [4] by pressing together the hinge grip pressure points [5] on the frame base [4].

Detach the forehead pad [1] from the forehead frame.

Detach the tubing connector [21] from the exhalation system air junction [8].

Remove the exhalation system insert tube [15] from the air junction [8].

Push the mask cushion [12] backwards out of the frame base.

Detach the exhalation system air junction [8] from the frame base by turning it counterclockwise to release it, and then pulling it out.

Pull off the connector cover [9] from the external pressure measurement connector [10].

Assembly

Always wash your hands before assembling the components after cleaning. You will find information on cleaning the nasal mask in the "Cleaning in the Home Environment" section. Follow the steps shown in Figures A to H on the inside front cover.

Fig. B: Press together the hinge grip pressure points [5] so that the forehead frame [3] can be pushed into the guide rail of the frame base [4].

Fig. D: Press the exhalation system air junction [8] from above into the round opening in the frame base. Insert the two different-sized catches into the corresponding slots. Secure the air junction by turning it clockwise to the limit stop.

Fig. E: Push the round opening of the mask cushion [12] over the air junction to the limit stop. There is a catch under the mask cushion; this has to lock into the frame [Fig. E-a]. Make sure that the tongue joints [13] snap into the matching groove joints [14] on the sides of the mask cushion [Fig. E-b].

Fig. F: It is easier to attach the mask cushion [12] first, and then attach the exhalation system insert tube [15]. Feed it through the mask cushion from behind and insert it from below into the exhalation system air junction to the limit stop.

Fig. H: Lay out the AeroFix II headgear [18] in front of you, so that the side with the washing instructions is facing away from you. Connect the longer straps to the upper slots on the mask frame and the shorter ones to the lower slots, and to the clip fastener.

Optional: Attach the middle of each upper strap of the headgear by the AeroFix plus tubing clip [19], available separately (REF 632 006).

Fitting the Nasal Mask

Always wash your face and hands before fitting the mask. Follow the steps shown in Figures I to M on the inside front cover.

Tip: Stand in front of a mirror when fitting your Silent Papillon nasal mask.

Fig. I: Loosen the straps by releasing the Velcro® fasteners. With the clip fastener open, pull the headgear over your head.

Fig. J: Close the clip fastener. Tighten the straps on each side and close the Velcro fasteners. Ensure that the upper straps are at the same height above each ear. The lower straps must be below the ears. The saddle part of the headgear should be centered at the back of your head.

Fig. L: (Optional) Secure the air tubing [20] over your head using the Velcro fastener of the AeroFix plus tubing clip [19], available separately.

Switch on your respiratory therapy device and get into bed. Check that the mask is fitted correctly.

Note: The mask should not pinch, and should not leave any pressure marks. No air should be able to escape between the skin and the mask cushion. Use the headgear to adjust the fit of your Silent Papillon nasal mask accordingly.

If necessary, you can adjust the angle between the frame base [4] and the forehead frame [3] to four different positions [22] [Fig. M]. The forehead pad [1] can be attached to the upper or lower attachment holes [2] on the forehead frame.

Cleaning in the Home Environment

For reasons of hygiene, the nasal mask must be cleaned every day.

Wash your hands before cleaning the nasal mask, and always clean and rinse the mask with drinking-quality water. Never use soap, chlorinated, corrosive, or abrasive products, or cleaning agents with a high alcohol content, since these may damage the material. Never use metal brushes or steel wool.

Cleaning

Clean all components of the nasal mask daily after use with a warm (approx. 30°C), mild dishwashing solution for about 5 minutes.

For effective cleaning, take the mask apart as described in the "Disassembly of the Nasal Mask" section. Remove any deposits in the air junction [8]. For optimum cleaning, the mask cushion can be turned inside out.

Then rinse all parts thoroughly in drinking-quality water to remove all remaining traces of dishwashing solution.

The AeroFix II headgear [18] and AeroFix plus tubing clip [19] should be cleaned once weekly. Detach the headgear and tubing clip from the nasal mask and close the Velcro fasteners.

The headgear may be washed by hand or in the washing machine at 30°C. Dry the headgear on slow spin, or drip dry.

Caution! The headgear is sensitive to heat, and must not be ironed.

Drying

For reasons of hygiene, it is important to dry the nasal mask thoroughly. Wipe the cleaned components dry with a clean, lint-free cloth.

Caution! Ensure that the nasal mask is not exposed to direct sunlight or placed on a heater, since this could damage the mask material.

Thermal Disinfection

Disinfection of the nasal mask in the home environment is generally not required. Disinfection of the mask may be necessary where infectious diseases are present, in the case of a known immune deficiency, or where directed by a physician. The purpose of the disinfection process is to kill pathogens. For disinfecting your nasal mask, we recommend the use of a commercial baby bottle disinfecter.

Warning! Burn hazard. Always comply strictly with the contents of the disinfecter instruction manual.

The nasal mask must be disassembled, cleaned, rinsed, and dried before disinfecting.

Always disinfect with distilled water, to prevent any lime deposits from building up on the surface of the nasal mask. The water vapor must reach all parts of the nasal mask. Allow the nasal mask to cool before removing it from the baby bottle disinfecter. After disinfection, the nasal mask must be dried once again.

The nasal mask can be thermally disinfected up to 60 times using a baby bottle disinfecter.

Functional Test

Test the correct functioning of the nasal mask after each cleaning or disinfection operation. Inspect the nasal mask for any defects or changes in the condition of the materials. A defective mask must always be replaced.

Assuming the cleaning procedure described above is followed, the Silent Papillon nasal mask, AeroFix II headgear, and AeroFix plus tubing clip have a useful life of one year.

Storage

Keep the nasal mask in a dry, dust-free location protected from direct sunlight until next required.

Cleaning, Disinfection, and Sterilization in the Hospital Environment

Information on cleaning, sterilization, and disinfection procedures for the Silent Papillon nasal mask in the hospital environment is available on request from MAP Medizin-Technologie GmbH (REF 560 081).

Disposal

The Silent Papillon nasal mask, AeroFix II headgear, and AeroFix plus tubing clip contain no toxic substances. Dispose of the nasal mask in accordance with applicable national laws and regulations. Except where stipulated otherwise, before being taken to a recycling center the nasal mask should be cleaned in accordance with the procedures described in this Instruction Manual.

Statutory Warranty

A two-year statutory warranty applies for manufacturing defects. The statutory warranty does not cover normal wear and tear. For further details, see the MAP Medizin-Technologie GmbH General Terms and Conditions of Business.

MAP shall not be liable for any impairments to the safety, reliability, or performance of the product caused by:

- use other than that specified in the Instruction Manual
- use in conjunction with other products not designed for this purpose
- any technical alterations made to the product.

The nasal mask must always be cleaned and disinfected before being returned for servicing. On completion of the above procedure, dry the mask and pack it in a sealable padded mailing bag.

Symbols

Attention: consult accompanying documents (Instruction Manual)

Warning: failure to comply with the instructions could pose a health risk.

REF Part number

Lot number

Date of manufacture (month, year)

Washing temperature 30 °C

Do not use chlorine bleach

Do not iron

Do not tumble dry

Technical Data

Standards

CE labeling in accordance with EC directive 93/42/EEC, class IIa

Quality Management: development, manufacture, sale, and service certified in accordance with ISO 13485:2003 quality management principles

EN ISO 17510-2

(Sleep apnea respiratory therapy, Part 2: masks and application accessories)

Validated Cleaning and Disinfection

The general suitability of the Silent Papillon nasal mask for effective cleaning and disinfection using the procedures specified in this Instruction Manual has been established by an independent accredited test laboratory.

Flow Resistance

Pressure drop measured with mask cushion size 2
at 50 l/min = max. 0.1 hPa
at 100 l/min = max. 0.4 hPa

Tubing Connection

Ø 22 mm for elastomer tubing

Weight

Approx. 92 g

Therapeutic Pressure

4 to 20 hPa

Storage Temperature

-20 °C to +60 °C

Operating Temperature

+5 °C to +40 °C

Materials Used

Mask (latex-free): silicone, PSU, PC

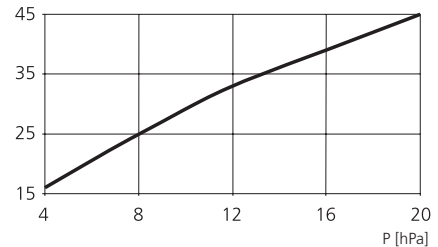
Headgear (neoprene-free): nylon, lycra, spandex, polyurethane, polyester

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