Chin Restraint Nasal Mask Accessory


Reorder No. 168140/1 05 05

USER'S GUIDE

INTENDED USE

The intent of the Chin Restraint is to restrict the mouth from opening during Positive Airway Pressure (PAP) treatment and minimise air loss through the mouth. It can be used in conjunction with a wide range of mask and pillow systems.

If possible, have the Chin Restraint initially fitted by staff at your sleep disorders center or by your equipment supplier.

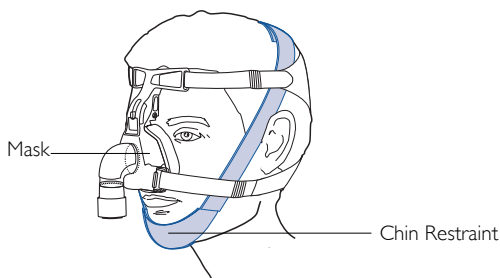FITTING THE CHIN RESTRAINT

The Chin Restraint can be fitted either before or after fitting the mask.

- 1 Fit the Chin Restraint over your chin so that the edge with the stitched 'V' mark is closest to your bottom lip. Bring the straps up past your cheeks and fasten the Velcro® at the top of your head.
- 2 If necessary, adjust the straps to provide sufficient support to your chin.

If you require a more secure fit, thread the Chin Restraint under your headgear strap as follows:

- 1 Fit the mask and headgear, then fit the Chin Restraint over your chin.
- 2 Pass the Chin Restraint strap on your right-hand side (without the Velcro tab) under the headgear strap at the top center of your head.
- 3 Then pass the other strap (with the Velcro Tab) across the top of both the headgear and Chin Restraint straps. Fasten the Velcro Tab to lock both the Chin Restraint and headgear in position.

Note: *If you experience any difficulty with the fitting of the Chin Restraint, consult your physician or equipment supplier.*

CAUTION

- Do not position the Chin Restraint so that it obstructs your mouth.
- Do not position the Chin Restraint completely beneath the chin or across your upper throat.
- Do not overtighten the Chin Restraint.

CLEANING

When dirty, handwash in warm water using a mild detergent. Rinse thoroughly and drip dry.

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Manufactured by: ResMed Ltd | Elizabeth MacArthur Drive Bella Vista NSW 2153 Australia Tel: +61 (2) 8884 1000 or 1 800 658 189 (toll free) Fax: +61 (2) 8883 3114 Email: reception@resmed.com.au
ResMed Offices: ResMed Corp (US Designated Agent) 14040 Danielson Street Poway CA 92064-6857 USA Tel: +1 (858) 746-2400 or 1-800-424-0737 Fax: +1 (858) 746-2900 • **ResMed (UK) Ltd (EU Authorized Representative)** 65 Milton Park Abingdon Oxfordshire OX14 4RX UK Tel: +44 (1235) 862 997 Fax: +44 (1235) 831 336 • **ResMed SA** Parc de la Bandonnière 2 rue Maurice Audibert 69800 Saint-Priest France Tel: +33 (0) 4 37 251 251 Fax: +33 (0) 4 37 251 260 • **ResMed GmbH & Co. KG** Fraunhoferstraße 16, 82152 Martinsried Deutschland Tel: +49 (0) 89 / 99 01 - 00 or +49 (0) 800 2 777 000 (gebührenfrei) Fax: +49 (0) 89 / 99 01 - 10 55 • **ResMed Asia Pacific Ltd** North Ryde Australia • **ResMed Austria GmbH** Villach Österreich • **ResMed BV** Den Haag Nederland • **ResMed Finland** Espoo Suomi • **ResMed Hong Kong** Tsim Sha Tsui Hong Kong • **ResMed Japan** Tokyo Japan • **ResMed Malaysia Sdn Bhd** Kuala Lumpur Malaysia • **ResMed NZ Ltd** Auckland New Zealand • **ResMed Singapore Pte Ltd** Novena Square Singapore • **ResMed Spain SL** Madrid España • **ResMed Sweden AB** Trollhättan Sverige • **Labhardt AG** Basel Schweiz

US DESIGNATED AGENT: ResMed Corp

EU AUTHORIZED REPRESENTATIVE: ResMed (UK) Ltd

Velcro is a registered trademark of Velcro Industries B.V.

© 2005 ResMed Ltd.

CE
0120