
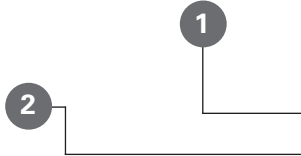


S9™ Visual Inspection—Compliance


Determining patient compliance with the S9

The ResMed S9™ displayed information meets Medicare criteria for visual inspection of adherence demonstrating the patient is using the device for four or more hours per night for 70% of nights in a consecutive 30-day period.

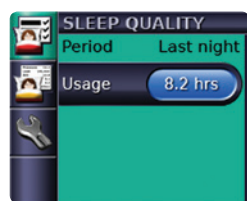


Access Advanced Menu

1. From the S9 **Home** screen, press the “i” (information) button
2. This will take you to the **Sleep Quality** screen
3. Turn the push dial counter-clockwise to access the **Sleep Report**

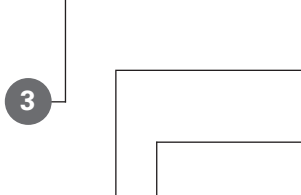




Select the proper time period

1. From the **Sleep Report** screen, press the push dial (**Period** should be highlighted)
2. Press the push dial again – **Period** should now be highlighted in yellow
3. Turn the push dial clockwise to change the period of time to **1 Month** (selects the last 30 consecutive days)
4. Press the push dial to access the usage report in months
5. Read the second line **Days > 4hrs**

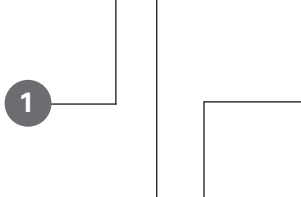

How to determine the number of days the patient has completed four or more hours of therapy in the last 30 days

1. From the **Sleep Report** screen, read the second line **Days > 4hrs**
2. Calculate the compliance percentage:
Divide the number of compliant days by the number of total of days:
(Days > 4 hours)/(days) Example: 21/30 days = 70% compliance

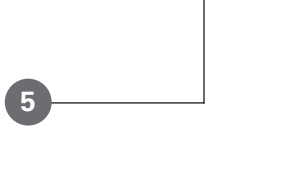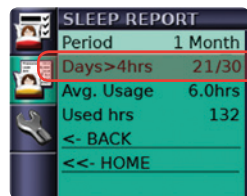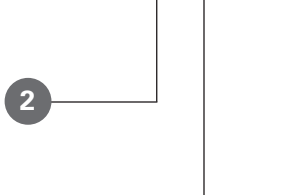

For additional information, please see the Medicare “PAP Devices for Treatment of OSA” Local Coverage Decision and FAQ documents available through the DME Medicare Administrative Contractors.

S9 Compliance Modules and Reporting

S9 compliance modules and reports provide you with the information you need to quickly identify causes of noncompliance, so you can focus on patients who require intervention.

ResScan™ Patient Management System

First 30 Days Report for individual patient records, allowing physicians and payors to easily identify when the patient has achieved compliance, their overall compliance percentage and their consecutive 30-day period of compliance

Wireless module

- Remote compliance monitoring
- Web-based monitoring device that transmits usage and summary data to an online database
- 24-hour access to patient information, allowing you to quickly intervene with patients who are experiencing difficulties at the start of therapy (eg, leak issues)

SD card

- Allows you to download data directly from the SD card into the patient management software
- 365 days of usage and summary data to determine overall compliance
- 30 days of detailed data to help you identify reasons for noncompliance (eg, excessive leak)
- Seven days of high-resolution data for a deep dive into complex patient cases

