## **VPAP™ Adapt**

POSITIVE AIRWAY PRESSURE DEVICE

## **H5i™**

HEATED HUMIDIFIER

## **ClimateLine™ / ClimateLine<sup>MAX</sup>™ / SlimLine™ / Standard**

AIR TUBING

## **Information Guide**

English • Français • Español • Português

### English

Please read the entire Information and Welcome Guides before using your device.

### **CAUTION**

**In the US, Federal law restricts this device to sale by or on the order of a physician.**

## **VPAP Adapt indications for use**

The S9 VPAP Adapt is indicated for the treatment of patients weighing more than 66 lb (30 kg) with obstructive sleep apnea (OSA), central and/or mixed apneas, or periodic breathing. The S9 VPAP Adapt is intended for home and hospital use.

## **VPAP Adapt contraindications**

Positive airway pressure therapy may be contraindicated in some patients with the following pre-existing conditions:

- severe bullous lung disease
- pneumothorax or pneumomediastinum
- pathologically low blood pressure, particularly if associated with intravascular volume depletion
- dehydration
- cerebrospinal fluid leak, recent cranial surgery, or trauma.

## VPAP Adapt adverse effects

Patients should report unusual chest pain, severe headache, or increased breathlessness to their prescribing physician. An acute upper respiratory tract infection may require temporary discontinuation of treatment.

The following side effects may arise during the course of therapy with the device:

- drying of the nose, mouth, or throat
- nosebleed
- bloating
- ear or sinus discomfort
- eye irritation
- skin rashes.

## H5i indications for use

The H5i is indicated for the humidification of the air delivered from a CPAP or bilevel device. The H5i is for use only as recommended by a physician. The H5i is intended for single patient re-use in the home environment and re-use in a hospital/institutional environment.

## H5i contraindications

The H5i is contraindicated for use with patients whose upper (supraglottic) airway has been bypassed.

## Troubleshooting

If there is a problem, try the following suggestions. If the problem cannot be solved, contact your equipment supplier or ResMed. Do not attempt to open the device or the H5i enclosure.

---

### Problem/Possible cause

### Solution

#### No display

Power is not connected.

Ensure the power cable is connected and the power outlet (if available) is on.

The DC plug is partially inserted into the back of the device or inserted too slowly.

Fully insert the DC plug.

The VPAP Adapt and H5i are not connected correctly.

Ensure that the VPAP Adapt and H5i are securely attached.

---

**Problem/Possible cause****Solution****Insufficient air delivered from the VPAP Adapt device**

Ramp time is in use.	Wait for air pressure to build up or change ramp time.
Air filter is dirty.	Replace air filter.
Air tubing is not connected properly.	Check air tubing.
Air tubing is blocked, pinched or punctured.	Unblock or free the air tubing. Check the air tubing for punctures.
Mask and headgear are not positioned correctly.	Adjust position of mask and headgear.
Incorrect air tubing selected.	If you are using the SlimLine, Standard or 3m air tubing ensure that you have the correct air tubing selected via the menu.
The H5i flip lid is not latched correctly.	Close the flip lid ensuring that it clicks into place.
The H5i flip lid seal is not fitted correctly.	Make sure the flip lid seal is facing the right way up and fitted securely.

**Device does not start when you breathe into the mask**

Breath is not deep enough to trigger SmartStart/Stop.	Take a deep breath in and out through the mask.
There is excessive leak.	Adjust position of mask and headgear.
<b>Note:</b> <i>If Leak Alert is enabled, an audible alert is activated and a high leak message is displayed.</i>	Air tubing not connected properly. Connect firmly at both ends.
SmartStart/Stop is disabled.	Enable SmartStart/Stop.
<b>Note:</b> <i>SmartStart/Stop is not available if Leak Alert is enabled.</i>	

<b>Problem/Possible cause</b>	<b>Solution</b>
<b>Device does not stop when you remove your mask</b>	
SmartStart/Stop is disabled.	Enable SmartStart/Stop.
<b>Note:</b> <i>SmartStart/Stop is not available if Leak Alert is enabled.</i>	
<b>SmartStart/Stop is enabled but the device does not stop automatically when you remove your mask</b>	
Incompatible mask system being used.	Only use equipment recommended by ResMed.
The patient is using a nasal pillows mask with a set pressure less than 7 cm H <sub>2</sub> O.	Disable SmartStart/Stop
<b>Pressure rises inappropriately</b>	
Talking, coughing or breathing in an unusual manner.	Avoid talking with a nasal mask on, and breathe as normally as possible.
Mask cushion is buzzing against the skin.	Adjust the headgear.
Cushion seated incorrectly causing excessive leak.	Adjust headgear or re-fit cushion.
<b>Displays error message: Heated tube fault, replace tube</b>	
Device has been left in a hot environment.	Allow to cool before re-use. Disconnect the power cord and then reconnect it to restart the device.
There is a fault in your ClimateLine or ClimateLine <sup>MAX</sup> heated air tubing.	Discontinue using your ClimateLine or ClimateLine <sup>MAX</sup> heated air tubing and contact your clinician/service provider. Use SlimLine, Standard or 3m air tubing in the interim.
<b>Displays error message: Humidifier fault, replace humidifier</b>	
Device has been left in a hot environment.	Allow to cool before re-use. Disconnect the power cord and then reconnect it to restart the device.

<b>Problem/Possible cause</b>	<b>Solution</b>
There is a fault in your H5i heated humidifier.	Discontinue using your H5i and contact your clinician/service provider.
<b>Displays error message: High temperature fault, refer to user manual</b>	
Device has been left in a hot environment.	Allow to cool before re-use. Disconnect the power cord and then reconnect it to restart the device.
Air filter is blocked.	Replace your air filter. Disconnect the power cord and then reconnect it to restart the device.
Air tubing is blocked.	Check your air tubing and remove any blockages. Disconnect the power cord and then reconnect it to restart the device.
Humidifier setting is too high, resulting in accumulation of water in the air tubing.	Turn the humidifier setting down and empty the water from the air tubing.
<b>Displays error message: Check ResMed 30/90W Power Supply Unit and fully insert the connector</b>	
The DC plug is partially inserted into the back of the device or inserted too slowly.	Fully insert the DC plug.
A non-ResMed power supply unit is connected to the device.	Remove the power supply unit and replace with a ResMed power supply unit.
The power supply unit is being covered by bedding.	Make sure that the power supply unit is free from bedding, clothes or other objects that could cover it.
<b>Displays error message: Tube blocked, please check your tube</b>	
Air tubing is blocked.	Check your air tubing and remove any blockages. Disconnect the power cord and then reconnect it to restart the device.
<b>Displays error message: High leak, please check system setup and all connections</b>	
There is excessive leak.	Adjust position of mask and headgear. Air tubing is not connected properly. Connect firmly at both ends.
The H5i flip lid is not latched correctly.	Close the flip lid ensuring that it clicks into place.

---

<b>Problem/Possible cause</b>	<b>Solution</b>
-------------------------------	-----------------

**Displays error message:** Please close H5i flip lid, attach tube and press any key.

H5i flip lid is not closed.

Close the flip lid ensuring that it clicks into place.

Air tubing is not connected properly.

Connect firmly at both ends.

**The following message is displayed on the LCD after you try to update settings or copy data to the SD card:** Card error, please remove SD card and contact service provider

SD card is not inserted correctly.

Ensure that the SD card is inserted correctly.

You may have removed the SD card before settings were copied to the device.

Reinsert the SD card and wait for the Home screen or the **"Settings updated successfully, press any key"** message to appear on the LCD.

**Note:** *This message only appears once. If you re-insert the SD card after you have updated your settings, the message will not be re-displayed.*

**The following message is NOT displayed on the LCD after you try to update the settings using the SD card:** Settings updated successfully, press any key

The settings were not updated.

Contact your clinician/service provider immediately.

**Water splashing on your face from the H5i**

The water tub is overfilled.

Check that the water level is below the maximum water level mark.

Condensation is forming in the air tubing and mask.

Turn the H5i or relative humidity setting down via the menu.

**Leaking water tub**

The water tub may be damaged or cracked.

Contact your service provider for a replacement.

The cleanable water tub is not assembled correctly.

Check for damage and reassemble the cleanable water tub correctly.

<b>Problem/Possible cause</b>	<b>Solution</b>
<b>Air feels too warm/cold in the mask</b>	
The temperature of the ClimateLine or ClimateLine <sup>MAX</sup> heated air tubing is set too high/low.	Turn up/down the heated air tubing temperature via the menu.
<b>General technical specifications</b>	
<b>Power supply</b>	<b>90W power supply unit</b> Input range: 100–240V, 50–60Hz, 110V, 400Hz nominal for aircraft use Typical power consumption: 70W (80VA) Maximum power consumption: 110W (120VA)
	<b>30W power supply unit</b> Input range: 100–240V, 50–60Hz, 110V, 400Hz nominal for aircraft use Typical power consumption: 20W (40VA) Maximum power consumption: 36W (75VA)
	<b>90W DC/DC converter</b> Nominal inputs: 12V, 24V Typical power consumption: 70W Maximum power consumption: 110W
<b>Environmental conditions</b>	Operating temperature: 41°F to 95°F (+5°C to +35°C)
	Operating humidity: 10 to 95% non-condensing
	Operating altitude: Sea level to 8,500' (2,591m)
	Storage and transport temperature: -4°F to 140°F (-20°C to +60°C) Storage and transport humidity: 10 to 95% non-condensing
<b>Aircraft use</b>	ResMed confirms that the device/s meets the Federal Aviation Administration (FAA) requirements (RTCA/DO-160G, section 21, category M) for all phases of air travel.

<b>Electromagnetic compatibility</b>	<p>Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for residential, commercial and light industry environments.</p> <p>It is recommended that mobile communication devices are kept at least 1 m away from the device.</p> <p>Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on <a href="http://www.resmed.com">www.resmed.com</a>, on the Products page under <b>Service and Support</b>. Click on the PDF file for your language.</p>
--------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

<b>IEC 60601-1 classification</b>	Class II (double insulation), Type BF
-----------------------------------	---------------------------------------

### **VPAP Adapt technical specifications**

<b>Pressure and flow state</b>	<p>Operating pressure range (measured at the mask): 4 to 20 cm H<sub>2</sub>O (CPAP); 4 to 25 cm H<sub>2</sub>O (ASV)</p> <p>Maximum single fault steady state pressure: 30 cm H<sub>2</sub>O - if pressure exceeded for &gt; 6 sec; 40 cm H<sub>2</sub>O - if pressure exceeded for &gt;1 sec</p> <p>Pressure measurement tolerance: ±0.5 cm H<sub>2</sub>O ± 4% of the measured reading</p> <p>Flow measurement tolerance: ±6 L/min or 10% of reading, whichever is greater, at 0 to 150 L/min positive flow</p>
<b>Mode reading</b>	<p><b>CPAP mode</b> Set Pressure: 4 to 20 cm H<sub>2</sub>O</p> <p><b>ASV mode</b> EPAP: 4 to 15 cm H<sub>2</sub>O; Max PS: 8 to 16 cm H<sub>2</sub>O; Min PS: 3 to 6 cm H<sub>2</sub>O</p>
<b>Physical</b>	<p>Nominal dimensions (L x W x H): 6.0" x 5.5" x 3.4" (153 mm x 140 mm x 86 mm)</p> <p>Weight: 1.84 lb (835 g)</p> <p>Housing construction: Flame retardant engineering thermoplastic</p> <p>Air outlet: 22 mm conical air outlet (complies with ISO 5356-1:2004)</p>
<b>Air filter</b>	<p>Hypoallergenic air filter: Acrylic and polypropylene fibers in a polypropylene carrier</p> <p>Standard air filter: Polyester non-woven fiber</p>

<b>Sound</b> DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871:1996	<b>Pressure level (CPAP mode)</b>	
	With SlimLine air tubing:	26 dBA with uncertainty of 2 dBA as measured according to ISO 17510-1:2007
	With Standard air tubing:	27 dBA with uncertainty of 2 dBA as measured according to ISO 17510-1:2007
	With either SlimLine or Standard air tubing and H5i:	28 dBA with uncertainty of 2 dBA as measured according to ISO 17510-1:2007
	<b>Power level (CPAP mode)</b>	
	With SlimLine air tubing:	34 dBA with uncertainty of 2 dBA as measured according to ISO 17510-1:2007
	With Standard air tubing:	35 dBA with uncertainty of 2 dBA as measured according to ISO 17510-1:2007
	With either SlimLine or Standard air tubing and H5i:	36 dBA with uncertainty of 2 dBA as measured according to ISO 17510-1:2007
<b>Supplemental oxygen</b>	Recommended maximum supplemental oxygen flow: 15 L/min	
<b>H5i technical specifications</b>		
<b>Temperature</b>	Maximum heater plate temperature: 150°F (65°C) Temperature cut-out: 165°F (74°C) Maximum gas temperature: ≤ 106°F (≤ 41°C)	
<b>Physical</b>	Dimensions (L x W x H): 6.0" x 5.7" x 3.4" (153 mm x 145 mm x 86 mm) Weight (water tub): Docking station and unfilled water tub 1.5 lb (0.67 kg) Weight (cleanable water tub): Docking station and unfilled water tub 1.7 lb (0.77 kg) Water capacity: To maximum fill line 380 mL	
<b>Materials</b>	Docking station: Flame retardant engineering thermoplastic, aluminium Cleanable water tub: Injection molded plastic, stainless steel and silicone seal Water tub: Injection molded plastic, aluminium and thermoplastic elastomer	

## Air tubing technical specifications

Air tubing	Material	Length	Inner diameter
ClimateLine heated air tubing	Flexible plastic and electrical components	6'6" (2 m)	0.6" (15 mm)
ClimateLine <sup>MAX</sup> heated air tubing	Flexible plastic and electrical components	6'3" (1.9 m)	0.75" (19 mm)
SlimLine air tubing	Flexible plastic	6' (1.8 m)	0.6" (15 mm)
Standard air tubing	Flexible plastic	6'6" (2.0 m)	0.75" (19 mm)
3 m air tubing	Flexible plastic	9'10" (3.0 m)	0.75" (19 mm)
Heated air tubing temperature cut-out: ≤ 106°F (≤ 41°C)			

### Notes:

- The manufacturer reserves the right to change these specifications without notice.
- The temperature and relative humidity settings displayed for Climate Control are not measured values.
- Check with your clinician/service provider before using the SlimLine air tubing with devices other than the S9 or H5i.
- The electrical connector end of the heated air tubing is only compatible with the H5i air outlet and should not be fitted to the device or mask.

## Humidifier performance

The following settings have been tested at 71.6°F (22°C) ambient temperature:

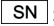
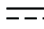
Mask pressure cm H <sub>2</sub> O	RH output %		Nominal system output AH <sup>a</sup> , BTPS <sup>b</sup>	
	Setting 3	Setting 6	Setting 3	Setting 6
3	90	100	10	18
10	95	100	11.5	21
20	95	100	11	18
25	100	100	12	13.5

a. AH - Absolute Humidity in mg/L.

b. BTPS - Body Temperature Pressure Saturated.

## Symbols

The following symbols may appear on your product or packaging.

 Caution;  Read instructions before use;  Drip proof;  Not drip proof;  Type BF equipment;  Class II equipment;  Start/Stop;  Manufacturer;  European Authorized Representative;  European RoHS;  Batch code;  Catalogue number;  Serial number;  Direct current;  Lock/unlock;  China pollution control logo 1;  China pollution control logo 2;  Not for use on more than one patient.;  Maximum water level;  Disinfectable up to 200°F (93°C);  Use distilled or deionized water only;  Remove tub to fill;  Follow instructions for use; **Rx Only** Prescription only (In the US, Federal law restricts these devices to sale by or on the order of a physician.)

## Servicing

The VPAP Adapt and H5i devices are intended to provide safe and reliable operation when operated in accordance with the instructions provided by ResMed. ResMed recommends that the VPAP Adapt and H5i be inspected and serviced by an authorized ResMed Service Center if there is any sign of wear or concern with device function. Otherwise, service and inspection of the devices generally should not be required during the five year design life of the device.

## Limited warranty

ResMed Ltd (hereafter 'ResMed') warrants that your ResMed product shall be free from defects in material and workmanship from the date of purchase for the period specified below.

Product	Warranty period
<ul style="list-style-type: none"><li>• Mask systems (including mask frame, cushion, headgear and tubing)—excluding single-use devices</li><li>• Accessories—excluding single-use devices</li><li>• Flex-type finger pulse sensors</li><li>• Humidifier water tubs</li></ul>	90 days
<ul style="list-style-type: none"><li>• Batteries for use in ResMed internal and external battery systems</li></ul>	6 months
<ul style="list-style-type: none"><li>• Clip-type finger pulse sensors</li><li>• CPAP and bilevel device data modules</li><li>• Oximeters and CPAP and bilevel device oximeter adapters</li><li>• Humidifier cleanable water tubs</li><li>• Titration control devices</li></ul>	1 year
<ul style="list-style-type: none"><li>• CPAP, bilevel and ventilation devices (including external power supply units)</li><li>• Humidifiers</li><li>• Battery accessories</li><li>• Portable diagnostic/screening devices</li></ul>	2 years

This warranty is only available to the initial consumer. It is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This Limited Warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organization that has not been expressly authorized by ResMed to perform such repairs; and c) any damage or contamination due to cigarette, pipe, cigar or other smoke.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty replaces all other expressed or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have resulted from the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

## **GENERAL WARNINGS**

- Use the device only for the intended use as described in this manual.
- Do not open the device or the H5i enclosure. There are no user serviceable parts inside. Repairs and servicing should only be performed by an authorized ResMed service agent.
- Use the device only as directed by your physician or healthcare provider.
- If you notice any unexplained changes in the performance of the device, if it is making unusual or harsh sounds, if the device or the power supply are dropped or mishandled, if water is spilled into the enclosure, or if the enclosure is broken, discontinue use and contact your ResMed Service Center.
- Beware of electrocution. Do not immerse the device, H5i docking station, power supply or power cord in water. Always unplug the device before cleaning and make sure they are dry before plugging them back in.
- Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps or scented oils to clean the device, H5i or air tubing. These solutions may cause damage and reduce the life of these products.
- Follow all precautions when using supplemental oxygen.
- Explosion hazard—do not use in the vicinity of flammable anesthetics.
- Only use the ResMed 90W or 30W power supply unit. Use the 90W power supply unit to power the system comprising the device, H5i, air tubing, DC/DC converter and battery pack. The 30W power supply unit is designed to power the device only and recommended for travelling.
- Make sure the power cord and plug are in good condition and the equipment is not damaged
- Keep the power cord away from hot surfaces.
- Do not drop or insert any object into any tubing or opening.
- Do not leave long lengths of air tubing around the top of your bed. It could twist around your head or neck while you are sleeping.
- Advice contained in this manual should not supersede instructions given by the prescribing physician.

## DEVICE WARNINGS

- Do not use the device if there are obvious external defects or unexplained changes in performance.
- Only ResMed air tubing and accessories should be used with the device. A different type of air tubing or accessory may alter the pressure you actually receive, reducing the effectiveness of your treatment. Do not use electrically conductive or antistatic air tubings.
- Blocking the air tubing and/or air inlet of the device while in operation could lead to overheating of the device.
- If your clinician has told you to use the SD card to update the settings on your device and the “Settings updated successfully” message does not appear, contact your clinician immediately.
- Do not connect a device to the module connector port unless instructed to do so by the health care provider or physician. Only ResMed products are designed to be connected to the module connector port. Connecting other devices could result in injury, or damage to the device.
- The device should only be used with masks (and connectors<sup>1</sup>) recommended by ResMed, or by a physician or respiratory therapist. A mask should not be used unless the device is turned on. Once the mask is fitted, ensure that the device is blowing air. The vent hole or holes associated with the mask should never be blocked.

**Explanation:** The device is intended to be used with special masks (or connectors) which have vent holes to allow continuous flow of air out of the mask. When the device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent holes. However, when the device is not operating, insufficient fresh air will be provided through the mask, and the exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This applies to most models of CPAP or bilevel devices.

- In the event of power failure<sup>2</sup> or machine malfunction, remove the mask.
- The S9 VPAP Adapt device can be set to deliver pressures up to 25 cm H<sub>2</sub>O. In the unlikely event of certain fault conditions, pressures up to 40 cm H<sub>2</sub>O are possible.
- Oxygen flow must be turned off when the device is not operating, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.

---

<sup>1</sup> Ports may be incorporated into the mask or in connectors that are near the mask.

<sup>2</sup> During partial (below rated minimum voltage) or total power failure, therapy pressures will not be delivered. When power is restored, operation will recommence with no change to settings.

- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame.

### **HUMIDIFIER WARNINGS**

- The H5i is designed only for use with S9 Series devices. Do not use the H5i with any other device.
- Do not operate the H5i if it is not working properly or if any part of the device or H5i has been dropped or damaged.
- The H5i should only be used with air tubing or accessories recommended by ResMed. Connection of other delivery tubes or accessories could result in injury, or damage to the device.
- Water must be prevented from entering the device
- Do not overfill the water tub as water may enter the device and air tubing.
- Take care when handling your H5i as the water/water tub may be hot. Allow 10 minutes for the heater plate and any excess water to cool.
- The H5i docking station should only be connected or disconnected when the water tub is empty.
- Make sure that the water tub is empty before transporting the H5i.
- Do not operate the H5i on an aircraft as water may enter the device and air tubing during turbulence.
- Always place the H5i on a level surface below the level of the user to prevent the mask and air tubing from filling with water.
- The H5i should be used with caution for users who are unable to remove the mask for themselves.
- If liquids are inadvertently spilled into or on the H5i docking station, unplug the device from the power outlet. Disconnect the docking station from the device and allow the docking station to drain and dry before re-using.
- Care must be taken to prevent the spilling of water on the device as this may cause the ingress of water into the device case.

### **AIR TUBING WARNINGS**

- Do not use the air tubing if there are any visible signs of damage.
- Do not wash the air tubing in a washing machine or dishwasher.
- The ClimateLine/ClimateLine<sup>MAX</sup> is designed only for use with the H5i.

## **DEVICE CAUTIONS**

- Be careful not to place the device where it can be bumped or where someone is likely to trip over the power cord.
- If you put the device on the floor, make sure the area is free from dust and clear of bedding, clothes or other objects that could block the air inlet or cover the power supply unit.
- Make sure the area around the device is dry and clean.
- Do not hang the air tubing in direct sunlight as it may harden over time and eventually crack.
- Do not wash the air filter. The air filter is not washable or reusable.
- The air filter cover protects the device in the event of accidental liquid spillage. Ensure the air filter and air filter cover are fitted at all times.
- At low pressures, the flow through the exhalation ports of your mask may not clear all exhaled gas from the air tubing. Some rebreathing may occur.
- The airflow for breathing produced by the device can be as much as 11°F (6°C) higher than the temperature of the room. Caution should be exercised if the room temperature is warmer than 90°F (32°C).

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© 2011 ResMed Ltd. 368433/2 2011-11
S9, H5i, ClimateLine, SlimLine, SmartStart and VPAP are trademarks of ResMed Ltd and S9, ClimateLine, SlimLine, SmartStart and VPAP are registered in U.S. Patent and Trademark Office.