RESMED

Ultra Mirage™ NON-VENTED FULL FACE MASK

Clinical Guide

English • Français • Español • Português

608140/2 06 11
Ultra Mirage NVFFM
CLINICAL
AMER ML

ResMed Ltd (Manufacturer) 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia
ResMed Corp (US Designated Agent) 14040 Danielson Street Poway CA 92064-6857 USA
ResMed (UK) Ltd (EU Authorized Representative) 65 Milton Park Abingdon Oxfordshire OX14 4RX UK
ResMed Offices Australia, Austria, Brazil, Finland, France, Germany, Hong Kong, Japan, Malaysia, Netherlands, New Zealand, Singapore, Spain, Sweden, Switzerland, UK, USA (see www.resmed.com for contact details).

Ultra Mirage Non-vented Full Face Mask

Protected by patents: AU 710733, AU 741003, AU 766623, AU 775051, CA 2261790, DE 29724224, EP 0956069, EP 1187647, EP 1187650, EP 1479406, JP 3686609, NZ 513052, NZ 526165, NZ 526166, NZ 526168, US 6112746, US 6119693, US 6357441, US 6374826, US 6463931, US 6513526, US 6532961, US 6581602, US 6634358, US 6701927, US 6860269, US 6871649, US 7069933. Other patents pending. Protected by design registrations: AU 157902, AU 302719, CN 200330102703.8, CN 200330102704.2, EP 098348, JP 1212687, JP 1214582, US D493885, US D519207.

Ultra Mirage and Mirage are trademarks of ResMed Ltd and Mirage is registered in U.S. Patent and Trademark Office.

Dove is a trademark of Unilever. CIDEX and STERRAD are trademarks of Johnson & Johnson. Alconox is a trademark of Alconox Inc.

A Mask Parts / Composants du masque / Piezas de la mascarilla / Peças da máscara

Forehead support pad / Tampon du support frontal / Almohadilla del apoyo para la frente / Almofada do suporte da testa

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
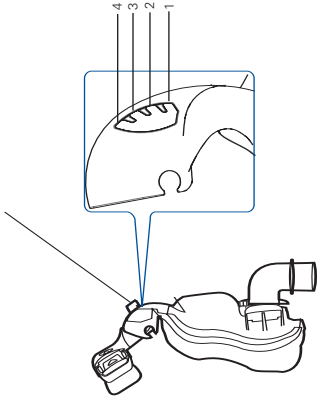
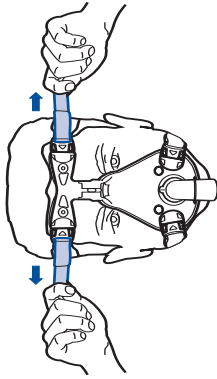
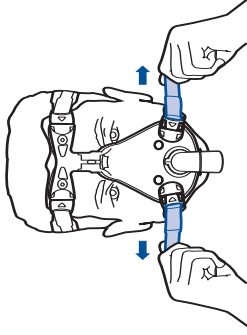
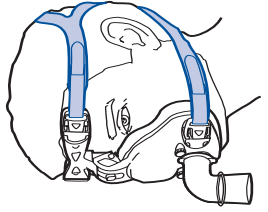
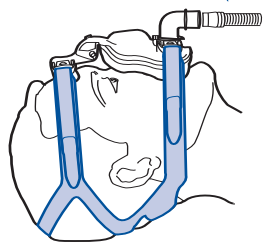
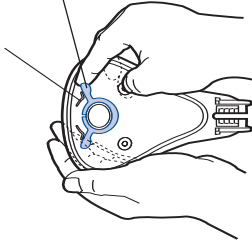
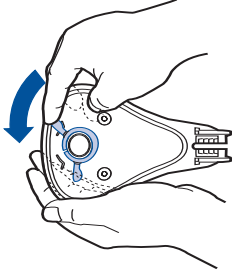
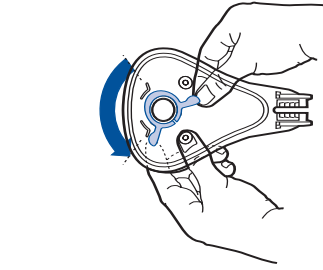
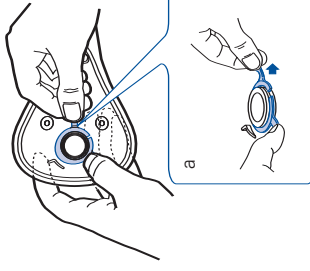
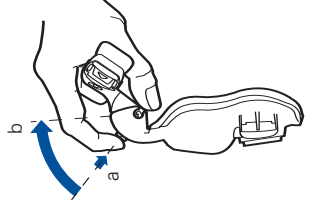
C-6

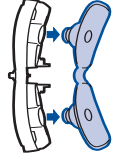© 2006 ResMed Ltd.

Ultra Mirage™
NON-VENTED FULL FACE MASK

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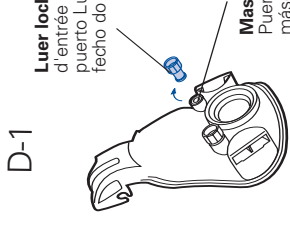
Note: For complete instructions, read the text section in conjunction with the illustrations on this sheet.

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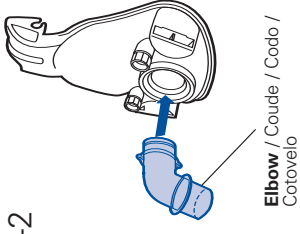
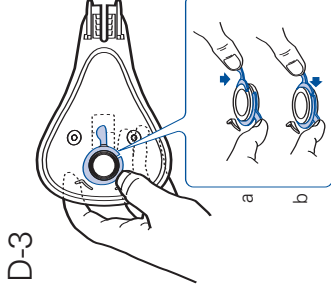
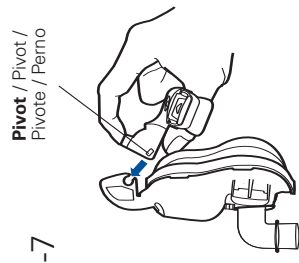
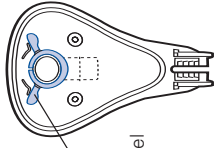
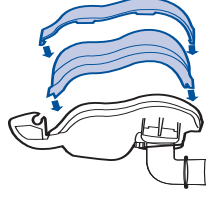
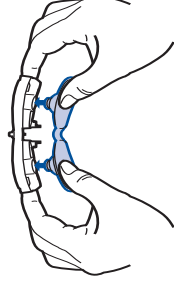
Ultra Mirage™

NON-VENTED FULL FACE MASK

The Ultra Mirage NV (non-vented) Full Face Mask is a non-invasive device used for channelling air flow with or without supplemental oxygen.

Intended Use

The Ultra Mirage NV Full Face Mask is intended to be used with active-exhaust-valve ventilator systems, to provide ventilatory assistance to patients with respiratory insufficiency and respiratory failure.

The Ultra Mirage NV Full Face Mask is to be used on adult patients (>66 lb/30 kg), requiring non-life-support ventilatory assistance.

The Ultra Mirage NV Full Face Mask is intended for single-patient re-use in the home or multipatient re-use in the hospital/institutional environment.

CAUTION

In the USA, federal law restricts this device to sale by, or on the order of, a physician.

WARNINGS AND CAUTIONS

- The Ultra Mirage NV Full Face Mask, must not be used without qualified supervision for patients who are unable to remove the mask by themselves.
- This mask is not for use on patients with impaired laryngeal reflexes or other conditions predisposing to aspiration in the event of regurgitation or vomiting.
- The mask should not be worn unless the ventilator system is turned on and operating properly.
- At a fixed rate of supplemental oxygen flow (if used), the inhaled oxygen concentration will vary depending on the pressure settings, patient breathing pattern, mask size selection and the mask leak.

- If oxygen is used with the ventilator, the oxygen flow must be turned off when the ventilator is not operating.
Explanation: When the ventilator is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the ventilator enclosure. Oxygen accumulated in the ventilator enclosure will create a risk of fire. This warning applies to most types of ventilators.
- Please refrain from smoking while oxygen is in use.

About the Mask

- The mask system does not contain latex, PVC or DEHP materials. If the patient has ANY reaction to any part of the mask system, discontinue use and investigate the possible causes.
- ResMed has tinted the Ultra Mirage NV Full Face Mask elbow blue to identify it as a non-vented mask for use with active exhaust non-invasive ventilator systems only.
- The mask has a standard female 22 mm tapered fitting (ISO5356-1) for external connection to the ventilator.
- Refer to the section “Technical Specifications” of this guide for information about mask leak, dead space and resistance.

CAUTION

The mask must be cleaned and disinfected/sterilized if it is used between patients. Refer to the section “Cleaning the Mask between Patients” for details.

Parts of the Mask

See section A on the illustrations sheet.

Fitting the Mask

Note: Refer to the *relevant ventilator system operating instructions for detailed information about connecting the mask.*

See section B on the illustrations sheet.

The Ultra Mirage NV Full Face Mask is supplied fully assembled.

- 1 Move the forehead support tab to Position 3. (Refer illustration **B-1** for location of Positon 3.)
- 2 Disconnect one lower headgear clip. Place the mask on the face and pull the headgear over the patient’s head (**B-2**). Ensure that the upper straps pass across the forehead and the lower straps below the ears.
- 3 Reattach the headgear clip to the mask frame.
- 4 Gently tighten the upper headgear straps (**B-3**).

5 Gently tighten the lower headgear straps (B-4).

The pads of the forehead support should sit comfortably on the forehead when the mask is in place. If the pads are at an angle and not completely seated on the forehead, change the position of the forehead support tab.

Note: Do not overtighten the headgear straps as this may cause discomfort, cushion creases and mask leak. The Ultra Mirage NV Full Face Mask has been designed to provide a comfortable seal without the need to overtighten the headgear.

6 The fitted mask should look like this (B-5).

7 Connect the main air tubing to the elbow (B-6). Then connect the free end of the air tubing to the ventilator and turn the ventilator on. If the patient experiences air leaks around the cushion, it may be necessary to gently tighten the headgear straps.

8 If the patient is unable to get a proper seal:

- reposition the mask by lifting it and placing the cushion comfortably on the face.
- adjust the position of the forehead support tab by testing all four positions and finding the one that best prevents leaks.
- gently tighten the headgear straps.

Notes:

- If it is not possible to resolve any leaks, your patient may require another mask frame and/or another cushion size.
- As with all mask ventilation systems, significant leak may occur between the mask and the patient's face. The leak may vary depending on such factors as the patient's position and sleep stage. Variation in leak may significantly affect patient ventilation, potentially producing profound hypoventilation or hyperventilation. The magnitude of this effect depends on the ventilator's ability to compensate for leak.
- Leak or variation in the leak may cause mistripping of the ventilator.

Removing the Mask

To remove the Ultra Mirage NV Full Face Mask, disconnect one of the lower headgear clips and pull the mask and headgear up over the patient's head.

Note: The headgear clips are easy to locate and disconnect. They allow the quick removal of the mask from the face in the event of an emergency.

Disassembling for Cleaning

See section C on the illustrations sheet.

- 1 Detach the mask from the air tubing.
- 2 Disconnect the headgear clips to release them from the mask frame. It is not necessary to remove the headgear clips from the headgear.
- 3 Remove the port caps from the mask frame.
- 4 Detach the cushion assembly from the mask frame by depressing the cushion clip at the base of the mask frame and gently pulling it off. This releases the cushion and cushion clip.
Then, separate the cushion from the cushion clip.
- 5 To detach the elbow from the frame, lift (C-1) and push (C-2) the right wing of the elbow retainer over the right L-shaped ridge on the mask frame.
- 6 Rotate the elbow retainer to the position shown in C-3. The elbow retainer is now in the correct position for removal.
- 7 To remove the elbow retainer, hold one wing of the elbow retainer down, and firmly pull the other wing out (C-4a) and then up (C-4b) over the top of the elbow. Remove the elbow retainer and elbow from the mask frame.
- 8 To remove the forehead support from the mask, press the tab (C-5a) and move it upwards until it is possible to slide it off the mask frame (C-5b).
- 9 Pull firmly at the base of the forehead support pad plugs to release the pads from the forehead support (C-6).

Cleaning the Mask in the Home

Notes:

- Wash your hands before cleaning the mask.
- It is important to always handwash the headgear.
- The headgear may be washed without being disassembled from the headgear clips.

Daily/After each use

Handwash the mask components in warm (approx. 86°F/30°C) water using mild soap or detergent. Rinse the components well and allow them to air dry out of direct sunlight.

The mask components have been tested using Dove™ Unscented beauty bar.

Weekly

Handwash the headgear in warm (approx. 86°F/30°C) water using mild soap or detergent. Rinse well and allow it to air dry out of direct sunlight.

CAUTION

- Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps, or scented oils to clean any of the mask components. These solutions may cause damage and reduce the life of the product.
- Do not expose any part of the mask system or tubing to direct sunlight as it may cause the product to deteriorate.
- The headgear is sensitive to heat and must not be ironed.
- If any visible deterioration of a mask component is apparent (cracking, crazing, tears, etc), the mask component should be discarded and replaced.

Cleaning the Mask between Patients

All mask components can be disinfected/sterilized using validated procedures. The following table presents a summary of the validated procedures for disinfection/sterilization of the system. Refer to the website www.resmed.com/masks/sterilization/americas for details.

Methods	Pre-disinfection/ sterilization	Disinfection/Sterilization
Chemical	Detergent eg, Alconox™	Solution (any one) <ul style="list-style-type: none"> • Ortho-phthalaldehyde 0.55% (eg, CIDEX™ OPA) • Glutaraldehyde 3.4% (eg, CIDEX Plus™)
Thermal	Detergent eg, Alconox	Temperature-time (any one) <ul style="list-style-type: none"> • 158°F (70°C) for 100 minutes • 167°F (75°C) for 30 minutes • 176°F (80°C) for 10 minutes • 194°F (90°C) for 1 minute
STERRAD™	Detergent eg, Alconox	System STERRAD Sterilization System
Notes:		
<ul style="list-style-type: none"> • Testing by ResMed has shown that the components can withstand 15 cycles of disinfection/sterilization for the methods listed in the table. • When using detergents, disinfectants or sterilization agents, always follow the manufacturer's instructions. 		

To clean the headgear between patients use the method described in the previous section "Cleaning the Mask in the Home".

Reassembling the Mask

See section D on the illustrations sheet.

- 1 Twist the luer lock port caps onto the mask ports (D-1).
- 2 Hold the mask frame and the elbow firmly together (D-2).
- 3 To attach the elbow retainer, fit one side of the elbow retainer against the groove of the elbow. While holding one wing of the elbow retainer down (D-3a), press the other wing down until the elbow retainer clicks into place (D-3b).
- 4 Rotate the elbow retainer to its original position (D-4).
- 5 Check that you have fitted the elbow and elbow retainer correctly by firmly pulling the elbow away from the frame. If the elbow is fitted correctly, it should not separate from the frame.
If the elbow comes away from the frame, try refitting the elbow retainer as described in Steps 3 and 4. If the elbow still comes away from the frame when you pull on it, you may need to replace the elbow retainer with a new one.
- 6 Fit the grooved edge of the mask cushion to the mask frame. Fit the top section first, then work your way around the mask, pushing the cushion onto the frame, ensuring a good seal has been achieved.
Then push the cushion clip over the cushion and onto the mask frame, ensuring it clicks into place in all locations (D-5).
- 7 Attach the forehead pads to the forehead support by inserting the pad plugs into the two holes on the back of the forehead support (D-6). Push the pads firmly against the forehead support to ensure that the pads fit securely into the forehead support.
- 8 Connect the forehead support to the mask frame. Slide the pivots into the pivot slot on the mask frame (D-7). Depress the forehead support tab and move it into any of the tab positions.
- 9 Insert the headgear clips into the slots on the mask frame. The headgear label should face outward and be near the patient's neck when fitted.

Disposal

The Ultra Mirage NV Full Face Mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

Technical Specifications

When determining ventilator settings, and in particular the tidal volume for volume cycled ventilators, make appropriate allowances for mask dead space and leak.

Unintentional Mask Leak

The Unintentional Mask Leak comprises leak between the cushion-frame and elbow-frame interfaces.

Pressure (cm H ₂ O)	Maximum Leak (L/min)
4-20	6

Dead Space

Dead space is the empty volume of the mask up to the elbow.

The dead space of the mask varies according to cushion sizes but is less than 260 mL.

Standards

CE designation in accordance with EC directive 93/42/EEC, class IIa.
EN ISO 14971, ISO 5356-1, ISO 594-1, ISO 10993-1 and ISO 13485.

Operating Pressure

0 to 40 cm H₂O

Resistance

Drop in pressure measured
at 50 L/min: 0.5 cm H₂O
at 100 L/min: 1.5 cm H₂O

Dimensions

The dimensions of the mask (Medium) are:
H = 200mm, W = 100mm and D = 100mm

Symbols

System and Packaging

Caution, consult accompanying documents

Lot number

Part number

Temperature limitation

Humidity limitation

Does not contain latex

Clinical Guide

Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.

Ordering Information

Part No.	Item
<i>System</i>	
60637	UM NVFFM System <i>Small, Standard</i>
60636	UM NVFFM System <i>Small, Shallow</i>
60639	UM NVFFM System <i>Medium, Standard</i>
60638	UM NVFFM System <i>Medium, Shallow</i>
60641	UM NVFFM System <i>Large, Standard</i>
60640	UM NVFFM System <i>Large, Shallow</i>
<i>Replacement Parts</i>	
16604	Cushion, <i>Small, Standard</i> (1 pk.)
16671	Cushion, <i>Small, Shallow</i> (1 pk.)
16605	Cushion, <i>Medium, Standard</i> (1 pk.)
16672	Cushion, <i>Medium, Shallow</i> (1 pk.)
16606	Cushion, <i>Large, Standard</i> (1 pk.)
16673	Cushion, <i>Large, Shallow</i> (1 pk.)

Part No.	Item
16674	Cushion clip, <i>Small</i> (1 pk.)
16675	Cushion clip, <i>Medium</i> (1 pk.)
16676	Cushion clip, <i>Large</i> (1 pk.)
60678	Elbow (1 pk.)
60683	Elbow retainer (1 pk.)
60681	Luer lock ports caps (2 pk.)
60682	Luer lock ports caps (10 pk.)
60115	Headgear clip (2 pk.)
60116	Headgear clip (10 pk.)
60123	Forehead pad (1 pk.)
60124	Forehead pad (10 pk.)
60675	Mask Frame, <i>Small</i> (1 pk.)
60676	Mask Frame, <i>Medium</i> (1 pk.)
60677	Mask Frame, <i>Large</i> (1 pk.)
60673	Forehead Support (1 pk.)
60917	Headgear (1 pk.)

Limited Warranty

ResMed warrants that your ResMed mask system, including its components: mask frame, cushion, headgear, tubing and other accessories (“Product”) shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

For further details refer to the Warranty.

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