Release Notes: VPAP™ Tx Lab System Version 5.0

English

GLOBAL

New release—VPAP™ Tx Lab System Version 5.0

Version 5.0 of the VPAP Tx Lab System introduces:

- Real-Time graphs
- Easy-Breathe waveform is available in S-Mode*
- Terminology change: EPAP replaces EEP in ASV mode
- Removal of CSAD in CPAP mode
- Support for additional therapy devices in particular regions

All regions

Stellar 100

Stellar 150

Europe and Asia Pacific

AutoSet CS2

VPAP Adapt SV

VPAP S (S9)

VPAP ST (S9)

Auto 25 (S9)

AutoSet CS (S9)

VPAP Adapt (S9)

Japan

S9 Sleepmate AutoSet

S9 Sleepmate Elite

S9 Respond

AutoSet CS

VPAP Adapt SV

*Americas only

Upgrade details

Version 5.0 provides one easy-to-use upgrade package for EasyCare Tx, Tx Link, VPAP Tx and Tx Link Administrator.

Upgrade times

- EasyCare Tx—less than 5 minutes per computer.
- Tx Link and (if necessary) VPAP Tx—approximately 15 minutes in total. Network traffic may increase these times.

Upgrade steps

Before you upgrade, ensure all Tx Link and VPAP Tx devices are turned on and connected.

Complete these steps to upgrade your system.

1. Save the upgrade software **EasyCareTx_5_00.exe** to your computer.
The upgrade can either be downloaded from the internet or installed from a CD or Flash drive.
2. Upgrade EasyCare Tx.
 - Double-click on **EasyCareTx_5_00.exe**
 - Follow the installation wizard.
When EasyCare Tx is upgraded, Tx Link Administrator is launched to enable you to perform the remaining upgrade steps.
3. Upgrade Tx Link and (if necessary) VPAP Tx.
 - In Tx Link Administrator click an active **Upgrade** button.
A progress bar will display and a blue arrow will flash to indicate the device being upgraded. The Tx Link and (if necessary) VPAP Tx have been upgraded when a message indicates 'These devices are ready for use'.
 - Click **Upgrade** if an active Upgrade button remains displayed against other devices in Tx Link Administrator.

The upgrade is now complete for any devices with a current connection to this computer.

4. Upgrade any additional computers using EasyCare Tx.
 - Copy **EasyCareTx_5_00.exe** to each computer and upgrade EasyCare Tx (see step 2).
 - When Tx Link Administrator is open:
 - If you have *point-to-point connections*, an active **Upgrade** button will display indicating you need to upgrade available devices (see step 3).
OR
 - If you have a *shared networked connection*, there may be no active **Upgrade** button. The available devices were upgraded from another computer.

Notes:

- All devices must be upgraded to the correct versions, otherwise the devices may not connect to each other. EasyCare Tx v5.0 (SX503-0500) will connect to Tx Link v5.0 (SX493-0500) and VPAP Tx v5.0 (SX469-0500)
- Remember to upgrade the EasyCare Tx application on each computer that will connect to a Tx Link device.

Troubleshooting

Problem/Possible cause

Solution

Displays error message: Current EasyCare Tx version is not compatible with selected Tx Link.

The current EasyCare Tx version is not compatible with selected Tx Link.

If the selected Tx Link has an earlier software version than EasyCare Tx, run Tx Link Administrator and upgrade the Tx Link.

If EasyCare Tx is an earlier software version than Tx Link, upgrade EasyCare Tx.

Displays Connecting... message indefinitely

The selected VPAP Tx is not compatible with the current version of EasyCare Tx.

Run Tx Link Administrator and upgrade the VPAP Tx device.

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