

Mask/Device Compatibility List

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

Note: Not all products are available in all regions.

Full Face Masks

Device	Mirage Quattro™	Mirage Liberty™	Ultra Mirage	Hospital Full Face
S9 AutoSet™	Full Face	Full Face	Full Face	Full Face
S9 Elite™	Full Face	Full Face	Full Face	Full Face
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.			
S7 Elite	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Vantage™	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.			
S8 Elite™	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™	No setting required. Consult your clinician before changing to a different mask type.			
S8 AutoSet™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Elite™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™ II	No setting required. Consult your clinician before changing to a different mask type.			
VPAP™ III	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP Adapt SV™	FULL FACE	FULL FACE ¹	FULL FACE	FULL FACE
VPAP™ Auto 25	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP Malibu™	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ S VPAP™ ST	MIR FULL	MIR FULL	MIR FULL	MIR FULL

¹ SmartStop may not operate as described when using the mask with ResMed VPAP Adapt SV.

Nasal Masks

Device	Mirage™ SoftGel	Mirage Activa™ LT	Mirage Micro™	Ultra Mirage II	Mirage Activa™	Mirage Vista™	Hospital Nasal	Mirage Kidsta™
S9 AutoSet™	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal
S9 Elite™	Nasal	Nasal	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.							N/V
S7 Elite	STANDARD	STANDARD	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	N/V
S8 AutoSet Vantage™	ACTIVA	ACTIVA	STANDARD	ULTRA	ACTIVA	STANDARD	STANDARD	N/V
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.							N/V
S8 Elite™	ACTIVA	ACTIVA	STANDARD	ULTRA	ACTIVA	STANDARD	STANDARD	N/V
S8 Escape™	No setting required. Consult your clinician before changing to a different mask type.							N/V
S8 AutoSet™ II	ACTIVA	ACTIVA	STANDARD	ULTRA	ACTIVA	STANDARD	STANDARD	N/V
S8 Elite™ II	ACTIVA	ACTIVA	STANDARD	ULTRA	ACTIVA	STANDARD	STANDARD	N/V
S8 Escape™ II	No setting required. Consult your clinician before changing to a different mask type.							N/V
VPAP™ III	STANDARD	STANDARD	STANDARD	ULTRA	STANDARD	STANDARD	N/V	N/V
VPAP™ III ST-A	STANDARD	STANDARD	STANDARD	ULTRA	STANDARD	STANDARD	N/V	STANDARD
VPAP Adapt SV™	ACTIVA	ACTIVA	VISTA ¹	ULTRA	ACTIVA	VISTA	VISTA	N/V
VPAP™ Auto 25	ACTIVA	ACTIVA	STANDARD	ULTRA	ACTIVA	STANDARD	STANDARD	N/V
VPAP Malibu™	N/V	N/V	MERIDIAN	ULTRA	N/V	VISTA	MERIDIAN	N/V
VPAP™ S VPAP™ ST	ACTIVA	ACTIVA	STANDARD	ULTRA	ACTIVA	STANDARD	STANDARD	N/V

¹ SmartStop may not operate as described when using the mask with ResMed VPAP Adapt SV.

N/V = The mask has not been validated for this device.

Nasal Pillows

Device	Swift™ FX	Swift™ LT for Her	Swift™ LT	Mirage Swift™ II
S9 AutoSet™	Pillows	Pillows	Pillows	Pillows
S9 Elite™	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.			
S7 Elite	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8 AutoSet Vantage™	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.			
S8 Elite™	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	No setting required. Consult your clinician before changing to a different mask type.			
S8 AutoSet™ II	SWIFT	SWIFT	SWIFT	SWIFT
S8 Elite™ II	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™ II	No setting required. Consult your clinician before changing to a different mask type.			
VPAP™ III	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP Adapt SV™	N/V	N/V	N/V	N/V
VPAP™ Auto 25	SWIFT	SWIFT	SWIFT	SWIFT
VPAP Malibu™	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S VPAP™ ST	SWIFT	SWIFT	SWIFT	SWIFT

N/V = The mask has not been validated for this device.

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide.

Tango, S7, AutoSet, S8, S8 AutoSet Vantage, S8 Compact, S8 Elite, S8 Escape, S9, S9 AutoSet, S9 Elite, VPAP, VPAP Adapt SV, VPAP Malibu, Mirage, Ultra Mirage, Mirage Quattro, Mirage Liberty, Mirage Vista, Activa, Kidsta, Swift and Mirage Swift are trademarks of ResMed Ltd.

© 2010 ResMed Ltd. 1013348/4 10 01