


Pressure vs. Flow Rates for ResMed Masks

This table and corresponding graph show the airflow rates through mask vents at varying mask pressures.



| Pressure (cm H ₂ O) | Nasal Masks | | | | Nasal Pillows | | | Full Face Masks |
|--------------------------------|---|----------------|--------------------|------------------|-------------------------------|--|--|-----------------|
| | Mirage™ FX Mirage FX for Her Mirage SoftGel™ Mirage Activa™ LT Mirage Micro™ Mirage Activa | Mirage Kidsta™ | Pixi™ ¹ | Ultra Mirage™ II | Swift™ LT Swift LT for Her | Swift FX Swift FX for Her Mirage Swift™ II | Quattro™ FX ² Quattro FX for Her ² Mirage Quattro™ Ultra Mirage™ Mirage Liberty™ | |
| 4 | 19 | 21 | 20 | 18 | 21 | 20 | 22 | |
| 6 | 24 | 26 | 25 | 24 | 25 | 25 | 28 | |
| 8 | 28 | 30 | 29 | 29 | 28 | 29 | 32 | |
| 10 | 31 | 33 | 32 | 34 | 31 | 33 | 37 | |
| 12 | 34 | 36 | 38 | 38 | 33 | 37 | 41 | |
| 14 | 37 | 39 | 40 | 43 | 36 | 40 | 44 | |
| 16 | 40 | 42 | 42 | 46 | 38 | 43 | 48 | |
| 18 | 43 | 45 | 46 | 50 | 40 | 46 | 51 | |
| 20 | 45 | 47 | 49 | 53 | 42 | 49 | 54 | |

Expected flow rates (Liters/min)

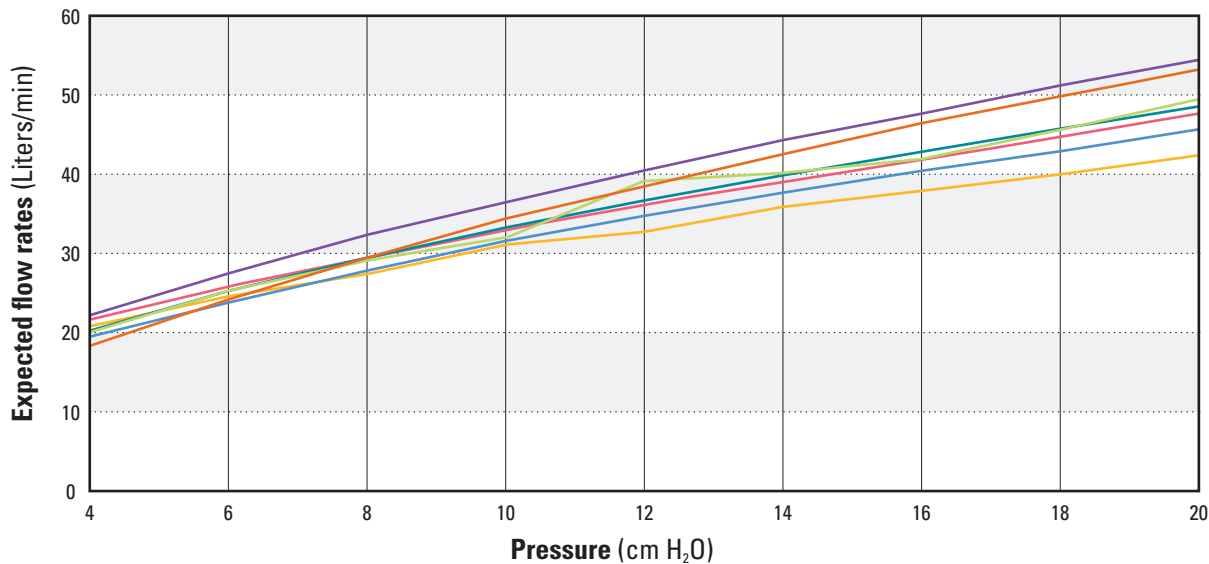

- Quattro FX, Quattro FX for Her, Mirage Quattro, Ultra Mirage, Mirage Liberty
- Ultra Mirage II
- Pixi
- Swift FX, Swift FX for Her, Mirage Swift II
- Mirage Kidsta
- Mirage FX, Mirage FX for Her, Mirage SoftGel, Mirage Activa LT, Mirage Micro, Mirage Activa
- Swift LT, Swift LT for Her

Note: The pressure flow data is the nominal specification curve, rounded to the nearest L/min. As a result of manufacturing variations, the vent flow rate may vary.

1 Pixi therapy pressure range 3-20 cm H₂O. See User Guide for pressure flow rates.
2 Quattro FX therapy range 4-30 cm H₂O. See User Guide for pressure flow rates.



Mask / Flow Generator Settings

The table below indicates which mask setting to use with your ResMed flow generator depending on which ResMed mask you are using.

| Patient Interface | S9™ Flow Generator Settings | S8™, S8 II Flow Generator Settings | Stellar™ 100 Settings | VPAP™ Flow Generator Settings | | | | | | |
|---|-----------------------------|------------------------------------|-----------------------|-------------------------------|---------------|-----------|---------------|--------------|------------------|-------------|
| | | | | VPAP III | VPAP III ST-A | Malibu | VPAP Adapt SV | VPAP Auto 25 | VPAP S / VPAP ST | VPAP Tx |
| Swift LT, Swift LT for Her, Mirage Swift II, Swift FX, Swift FX for Her Nasal Pillows | Pillows | Swift | Pillows | Mirage | Mirage | Swift | N/A | Swift | Swift | Pillows |
| Mirage FX, Mirage FX for Her Nasal Masks | Nasal | Standard | Nasal | Standard | Standard | Vista | Vista | Standard | Standard | Nasal |
| Mirage Micro Nasal Mask | Nasal | Standard | Nasal | Standard | Standard | Meridian | Vista | Standard | Standard | Nasal |
| Mirage Activa, Mirage Activa LT, Mirage SoftGel Nasal Masks | Nasal | Activa | Nasal | Standard | Standard | N/A | Activa | Activa | Activa | Nasal |
| Ultra Mirage II Nasal Mask | Nasal Ultra | Ultra | Ultra | Ultra | Ultra | Ultra | Ultra | Ultra | Ultra | Nasal Ultra |
| Mirage Kidsta, Mirage Micro for Kids Nasal Masks | N/A | N/A | Pediatric | N/A | Standard | N/A | N/A | N/A | N/A | N/A |
| Quattro FX, Quattro FX for Her, Ultra Mirage, Mirage Quattro, Mirage Liberty Full Face Masks | Full Face | Mir Full | Full Face | Mir Full | Mir Full | Full Face | Full Face | Mir Full | Mir Full | Full Face |
| Pixi | N/A | N/A | Pediatric | N/A | N/A | N/A | N/A | N/A | N/A | N/A |

For further information regarding ResMed products, please call Customer Service at 800.424.0737.