

Swing into Quattro FX

Achieving an effective seal is the key to successful therapy; but it needn't be complicated. Quattro FX introduces the two-step Swing Seal fitting technique.



1 Firstly position the cushion at an angle on the nasal bridge ...

2 ... then **swing** the lower half of the mask down against the face to **seal**.

Simply swing, and seal, for a secure seal and a good night's sleep.



Quattro™ FX
FULL FACE MASK

www.quattrofx.com

RESMED

Feature-rich, unobtrusive design

Quattro FX comes packed with ResMed's renowned attention to detail and innovation.



Complete System*

	Asia Pacific
Small	61763
Medium	61764
Large	61765

* Medium headgear included with all mask sizes.

For details, refer to the **Quattro FX Full Face Mask User Guide.**

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide. Quattro and Spring Air Technology are trademarks of ResMed Ltd and Spring Air Technology is registered in U.S. Patent and Trademark Office. © 2010 ResMed Ltd. Specifications may change without notice. 1014331/2 2010-11



RESMED

Less mask. **More you.**



Quattro™ FX
FULL FACE MASK

www.quattrofx.com

The unobtrusive alternative

Bringing light, unobtrusive design to a full face mask, Quattro™ FX provides an effective alternative to conventional full face masks and more choice in full face therapy.

With Quattro FX, therapy has never been easier

Benefits for you ...

It's as easy as ... Size. Swing. Seal

Quattro FX gives you the tools to achieve the best fit for your patients—first time, every time. The custom-made sizing tool and **Swing Seal** fitting technique simplify the sizing, fitting and sealing process.

Time and cost savings

The suspension-like combination of the Spring Air™ cushion and Spring frame means forehead support is unnecessary. Eliminating an additional point of adjustment, fitting Quattro FX is fast and straightforward ... saving you time, money and additional patient visits.

Therapy acceptance

The comfort, performance and feeling of openness that Quattro FX provides, help users accept and continue therapy ... at titration and beyond.

... and your patients

Quality and performance retained

New technologies in the Spring Air cushion and Spring frame offer the stability and angular adjustment provided by conventional forehead support. Quattro FX: quality and performance in a less obtrusive full face mask.

Freedom to be

Fewer points of contact with the face reduce the chances of skin irritation; a clear line of sight helps break down the physical and emotional barriers associated with full face therapy.

A good night's sleep

High-tech Spring Air cushion distributes pressure evenly and absorbs even the slightest user movements, so users can enjoy a good night's sleep, confident that their seal is safe.

Clear line of sight

and feeling of openness created by a full face mask that starts mid-way on the nasal bridge; a smaller frame also makes side-sleeping more comfortable

Built-in vent design

disperses air gently and quietly

Flexible, durable Spring frame

strong enough to stabilise the mask on the face, soft enough to accommodate user movement during sleep

360° elbow rotation

positions the tubing for ease of use


Swivel

offers a convenient way to connect and disconnect air tubing



New, tension-reducing headgear design

features a contoured crown strap that sits snug at the top of the head and avoids the sensitive area around the back of the neck

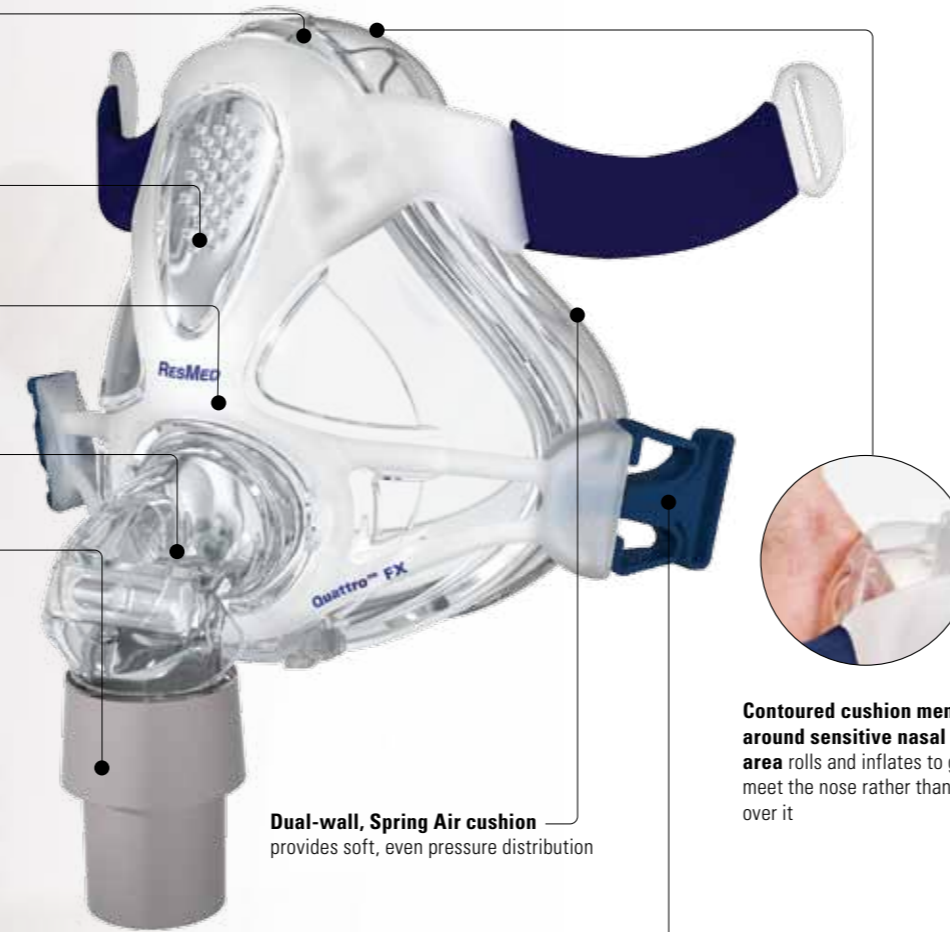

Dual-wall, Spring Air cushion

provides soft, even pressure distribution

Contoured cushion membrane around sensitive nasal bridge area rolls and inflates to gently meet the nose rather than stretch over it

Set-and-forget headgear clips

easy to grip for attaching and detaching; retain desired headgear settings

Fitting Guide

Prior to fitting your mask, remove the orange spring frame support from the rear of your mask and retain for future use.



- With both lower headgear clips released, position the cushion at an angle on the nasal bridge.



- Keeping the upper position of the mask in place, swing the lower half of the mask down against the face.



- Hold the mask firmly against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, clip the headgear into the lower part of the spring frame.



- Unfasten the Velcro®, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the Velcro.



- Repeat step 5 with the lower headgear straps.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

Tip One



- Ensure the headgear is correctly positioned on the head.

Tip Two



- Ensure the mask sits mid-bridge and does not encroach on the eyes.

Quattro™ FX
FULL FACE MASK

www.quattrofx.com