ResMed

Air10 DC/DC Converter 90W

User Guide

English | Deutsch | Français | Italiano | Español | Português | Nederlands | Svenska | Dansk | Norsk | Suomi |
Česky | Eesti | 简体中文 | 繁體中文 | 日本語

ENGLISH

The Air10™ DC/DC Converter 90W (model no. 370003) allows you to operate your device from a vehicle cigarette lighter socket (12V or 24V DC power source) in a car, boat, or other vehicle equipped with a suitable battery.

The Air10 DC/DC Converter 90W (model no. 370003) is specifically for use with the AirSense™ 10, AirCurve™ 10 and Lumis™ device.

Consult your equipment supplier for more information about this accessory.

Using the converter via a vehicle cigarette lighter socket

Refer to Illustration at the beginning of this document.

1. Before connecting the converter, make sure the converter switch (A) is off.
2. Position the converter on a level surface, with the ResMed logo facing up.
3. Plug the converter connector (B) into the device power inlet/DC input socket (C).
4. Remove the cigarette lighter from the vehicle cigarette lighter socket.
5. Plug the converter adaptor plug (D) into the vehicle cigarette lighter socket.
6. Switch on the converter. The light (E) will glow green when it powers on.

The light will flash continuously if the connection is not made, or if the device is in standby or power save mode

when the connection is made. If the battery is low, the light will glow amber. If the light does not operate, check that the converter switch (A) is on.

Notes:

- Some vehicles require the ignition key to be turned to the “accessories” position to provide power to the vehicle cigarette lighter socket.
- If you connect your device to a powered converter, the converter may power down. To resume power, turn the converter off and on again.
- The power cord is not replaceable.
- If the battery OR car, boat or other vehicle are not in good condition, there may not be enough reserve left in the battery to start the vehicle after use.
- The Air10/ Lumis device with the HumidAir™ humidifier attached uses considerably more power than the Air10/ Lumis device alone. Using the HumidAir humidifier will drain the battery more quickly.
- If the converter cuts power to your device when starting your car, boat or other vehicle, disconnect the converter from the vehicle. Wait a few seconds and then reconnect it.
- When the converter is not in use switch the converter off or disconnect it from the vehicle cigarette lighter socket to prevent battery discharge.
- If you need help, contact a ResMed representative, ResMed Technical Services or see www.resmed.com.

Cleaning

If necessary, wipe the outside of the converter with a clean, damp cloth.

⚠ WARNINGS

- Read instructions before use.
- Only use the converter when connecting to a 12V or 24V DC supply.
- Do not attempt to modify the converter. There are no user-serviceable parts inside the converter.

⚠ CAUTIONS

- The Air10 DC/DC Converter must only be used with AirSense 10/ AirCurve 10/ Lumis devices to ensure proper performance.
- The converter is intended for use on 12V or 24V negative ground systems (where the negative terminal of the battery is grounded). Using it on a 12V or 24V positive ground system may damage the converter and the vehicle's electrical system. If you are unsure whether you have a negative ground system, consult the car, boat or other vehicle manual.

- Make sure the converter is not covered during use to avoid overheating the converter.
- The converter is not intended for use on an aircraft.

Note: For any serious incidents that occur in relation to this product, these should be reported to ResMed and the competent authority in your country.

Technical specifications

Length (connector to connector)	9.8 ft (3 m)
Fuse rating	20A
Fuse type	Fast blow, low breaking capacity
Input	12V/24V  13/6.5A
Output	24V  3.75A
Peak power consumption	106W
Typical power consumption	51W
Operating temperature	+41°F to +95°F (+5°C to +35°C)
Operating humidity	10 to 95% non-condensing
Operating altitude	Sea level to 8,500' (2,591 m); air pressure range 1013 hPa to 738 hPa
Storage and transport temperature	-4°F to +140°F (-20°C to +60°C)
Storage and transport humidity	5 to 95% non-condensing

Electromagnetic compatibility

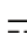
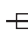
Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for residential, commercial and light industry environments.

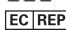
Ensure that wireless communications devices (such as mobile phones, wireless home network devices, cordless telephones and their base stations, and walkie-talkies) are kept at least 3.2 ft (1 m) away from the device.

Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on www.resmed.com, on the Products page under Service and Support.

Symbols which may appear on the device

 Warning or caution, consult accompanying documents;  Class II equipment; **IP21** Protection against insertion of fingers and against vertically dripping water; **IP22** Protection against insertion of fingers and will not become unsafe when it is exposed to vertically dripping water with the device tilted up to 15 degrees;

 Direct Current;  Fuse;  Manufacturer;

 European Authorized Representative;

 European RoHS;  Serial number;

 China pollution control;  Humidity limitation;

 Temperature limitation;  operating altitude;

 Atmospheric pressure limitation;  Importer;

 Medical device.

See symbols glossary at ResMed.com/symbols.

Environmental Information. The DC/DC converter should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment. If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

Limited warranty

ResMed warrants that your ResMed Air10 DC/DC Converter 90W shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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© 2022 ResMed. 3781073/1 2022-12
See ResMed.com for other ResMed locations worldwide. Air10, AirSense, AirCurve, Lumis, and HumidAir are trademarks and/or registered trademarks of the ResMed family of companies. For patent and other intellectual property information, see ResMed.com/ip.
ResMed Pty Ltd