## Tubing Wrap

UNIVERSAL AND SLIMLINE™

### User Instructions

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi •

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#### English

Tubing wraps are intended to reduce condensation and temperature loss in ResMed air tubing. They are made from nylon and polyester, and do not contain natural rubber latex.

There are two tubing wraps available:

- **Universal (blue):** for use with standard air tubing – 6' 6" (2 m) long, 0.75" (19 mm) inner diameter.
- **SlimLine (gray):** for use with SlimLine air tubing – 6' (1.8 m) long, 0.6" (15 mm) inner diameter.

#### Fitting

1. Fully unzip the tubing wrap and unfasten the Velcro™ strap.
2. Lay the tubing wrap down with the inside facing up.
3. Pass one end of the air tubing through the closed zipper end of the tubing wrap until the rubber cuff is through.
4. Hold the rubber cuff in position, then zip up the tubing wrap.
5. Fold the extended end of the zipper back against the tubing wrap and secure using the Velcro strap. Only the rubber cuffs should now be visible.

**Note:** If you have a *Universal (blue)* tubing wrap and you want to use it with *SlimLine* air tubing, you must complete this additional step between steps 3 and 4:

- At the open end of the tubing wrap, fold the extra length over by approximately 6" (15 cm) and secure using the Velcro.

#### WARNING

Do not use the tubing wrap if the room temperature is above 86°F (30°C). At higher temperatures the air temperature at the mask may be too hot.

#### Cleaning instructions

The tubing wrap should be hand washed or dry cleaned only. Wash the tubing wrap separately when new, as the colour may run. Do not bleach.

To wash the tubing wrap:

1. Wash in cold to warm water using a mild detergent.
2. Rinse well and drip dry.

**Note:** Hand washing your tubing wrap does not replace the need to clean your air tubing.

#### Symbols

The following symbols may appear on your tubing wrap or packaging.

Warning;  Follow instructions for use;

Manufacturer;  European Authorised

Representative;  Catalogue Number;  Batch Code;  Does not contain natural rubber latex.

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© 2012 ResMed Ltd. 368778/1 2012-01
Velcro is a trademark of Velcro Industries B.V.