
Standard Trolley



User Guide

English • Deutsch • Français • Italiano • Español • Svenska • Dansk • Norsk • Suomi


A

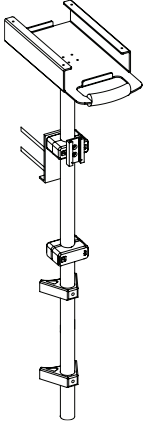
B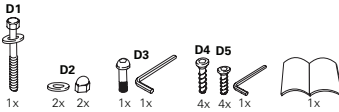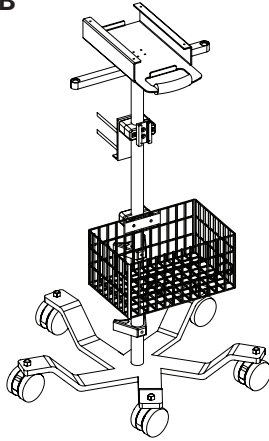

C

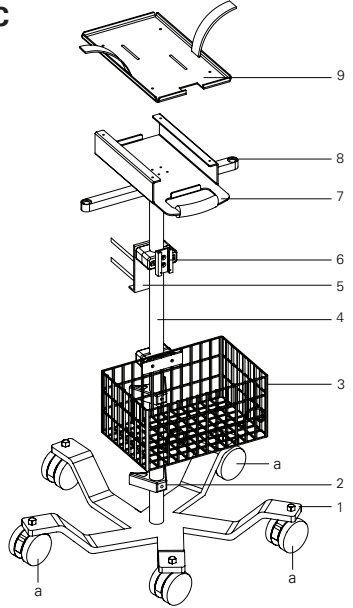D

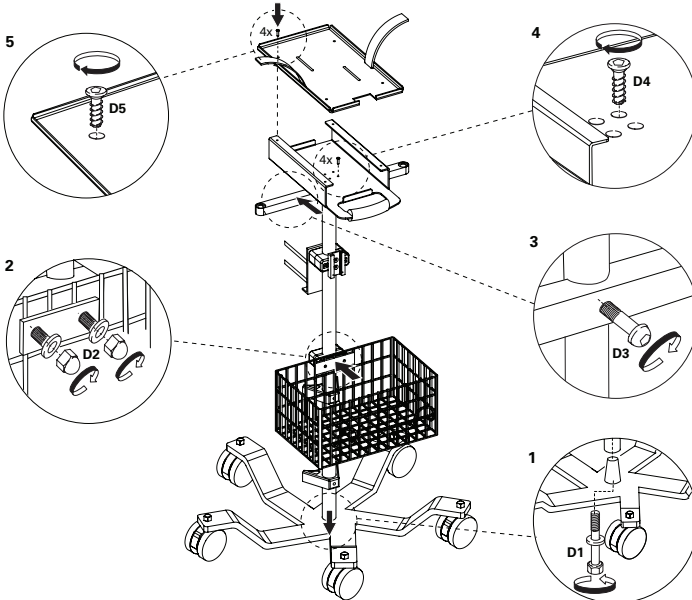Intended use

The ResMed Standard Trolley ('trolley') is designed for practical and convenient use to hold and move ResMed therapy devices and accessories between points of therapy in the home or hospital environment. It is intended to support the ventilation device, humidifier, battery pack, oxygen cylinder, circuit support arm, and other clinical devices as needed.

The trolley is validated for use with VPAP III, VPAP IV, Stellar, Elisée, and VS device series.

For a full list of compatible devices for the trolley, see the Ventilation Accessories on www.resmed.com on the **Products** page under **Ventilation Devices**. If you do not have internet access, please contact your ResMed representative.

Notes:

- For full details regarding correct use of these devices, please refer to your device user guide.
- Not all devices are available in all regions.

Components

Refer to A, B, and C on page 1.

Item	Description	Included	Part No.
A	Pole assembly	✓	-
B	Trolley system (includes items 1 to 8, assembly tools and user guide)	✓	24905
C	1 Wheeled base – with three lockable wheels (a)	✓	-
	2 Oxygen cylinder bracket	✓	-
	3 Utility basket	✓	-
	4 Pole	✓	-
	5 AC adapter bracket	✓	-
	6 Humidifier bracket	✓	-
	7 Battery shelf	✓	-
	8 Circuit support arm mount	✓	-
Ordered separately			
9	Device plate • For VPAP III, VPAP IV and Stellar devices • For Elisée and VS series		24913 24914

For other available accessories and spare parts, see the Ventilation Accessories Catalogue on www.resmed.com/ventilationaccessories.

Setting up

Refer to D on page 1.

1. Mount pole assembly to base while ensuring correct positions of the lockable wheels.
2. Attach basket to pole.
3. Attach circuit support arm mount to pole.
4. Mount battery shelf to pole, if not pre-mounted. Ensure battery shelf handle is on the same side as the basket.
5. Mount suitable plate on top of battery shelf. There are two plate designs suited to different ventilation devices.

Cleaning and maintenance

Wipe the trolley once a week or more often if required with a damp cloth and a mild cleaning solution.

Note: Solutions used in conjunction with the trolley may be corrosive. If solution spills on the trolley, clean immediately.

Disposal

The trolley does not contain any hazardous substances and may be disposed of with your normal household refuse.

Technical specifications

Height: 39.4" (1 m)

Base diameter: 22" (0.6 m)

Shipping weight: 37.5 lb (17 kg)

Load limit: 50 lb (23 kg)

Symbols

Follow instructions for use; Manufacturer;

Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.

General warnings and cautions

WARNING

- Read the entire guide before using the trolley.
- If this trolley is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the trolley.
- Do not place the trolley and device on steep slopes. Hold the trolley handle as you move the unit over power cords or any other obstacle to prevent accidental tipping.
- Before moving the trolley carrying a device integrated with a humidifier, make sure that the humidifier water tub is empty.
- Do not transport the trolley carrying therapy devices and accessories in a vehicle.
- Always ensure all three lockable wheels are locked to immobilise the trolley.

CAUTION

- Follow any additional safety and installation instructions that accompany the device that you are mounting on the trolley system.

Limited warranty

ResMed warrants that your ResMed trolley shall be free from defects in material and workmanship for a period of 90 days from the date of purchase by the initial customer. This warranty is not transferable.

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© 2018 ResMed Ltd. 248284/3 2018-11
VPAP and Stellar are trademarks of ResMed Ltd and VPAP is registered in U.S. Patent and Trademark Office. Elisée and VS III are trademarks of ResMed Paris SAS and Elisée is registered in U.S. Patent and Trademark Office.