

# RESMED

## H5i™ Cleanable Water Tub

### User Instructions

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • 简体中文 • 繁體中文

A

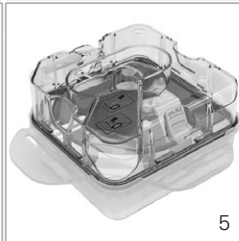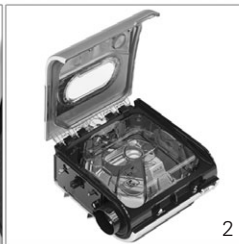You should regularly carry out cleaning and maintenance of the cleanable water tub as described in this section.

### Disassembling the H5i

Refer to illustration A.

1. Slide the latch.
2. Lift open the flip lid.
3. Remove the water tub.
4. Discard any excess water from the water tub.
5. Unclip all four side latches.
6. Pull apart the tub lid, plate and base.

### Daily cleaning

1. Wash the disassembled tub lid, plate and base in warm water using a mild detergent.
2. Rinse thoroughly in clean water and allow them to dry away from direct sunlight.

*Note: The disassembled tub lid, plate and base may also be washed in a dishwasher on the delicate or glassware cycle (top shelf only).*

### Monthly cleaning

1. Peel the flip lid seal from the flip lid and wash it in warm water using a mild detergent.
2. Wipe the exterior of the H5i with a damp cloth and mild detergent.

### Reassembling the H5i

Refer to illustration B.

1. Place the plate back onto the base.
2. Place the tub lid back onto the plate/base ensuring that the centre holes are aligned.
3. Clip all four side latches.
4. Fill the water tub and return the water tub to the H5i.
5. Close the flip lid ensuring that it clicks into place.

### Maintenance checklist

- ✓ Inspect the water tub and flip lid seal for wear and deterioration.
- ✓ Replace the water tub if any component is leaking or has become cracked, cloudy or pitted.

- ✓ Replace the flip lid seal if cracked or torn.
- ✓ Clean white powder deposits in the water tub by using a solution of one part household vinegar to 10 parts water.
- ✓ Inspect the maximum water level mark. If cleaning is required, remove by pinching and pushing out the locks from the plate. Wash in warm water using a mild detergent.

For more information on the H5i, see your Welcome and Information Guides. This documentation is also available to download from [www.resmed.com](http://www.resmed.com) on the Products page under Service & Support.

## **WARNING**

- Beware of electrocution. Do not immerse your device or power cord in water. Always unplug the device before cleaning and make sure it is dry before plugging it back in.
- Take care when handling your H5i. The water and the water tub base may both be hot. Allow 10 minutes for the heater plate and any excess water to cool.

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© 2017 ResMed Ltd. 368762/3 2017-11
ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia