

## Swift™ FX

NASAL PILLOWS SYSTEM

### Fitting Guide



1

- Holding the pillows at your nose, pull the headgear over your head.
- Headgear should sit between the ears and eyes.



2

- Adjust the top strap first.
- Pull the strap through the buckle one notch at a time.
- Do not overtighten.



3

- Adjust the back strap to fine-tune the fitting if necessary.
- Do not overtighten.
- The back strap can be positioned over or under the hair.



4

- Swift FX pillows are designed to provide a natural fit.
- Minor adjustment can help position the pillows at the most comfortable angle in the nose.



5

- Connect the Swift FX short tube to the device air tubing.
- Swift FX is now fitted and ready for use.



TIP

- With the device turned on, check that the nasal pillows are properly inflated. To do this, pull the nasal pillows away from the face and cheek, make sure they are inflated and reposition them in the nose.



#### OPTIONAL SOFT WRAPS

- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on your cheeks, with the fasteners on the outer side.

Note: For additional information, refer to the *Swift FX Nasal Pillows System User Guide*.