# RESMED

## Mirage™ SoftGel

NASAL MASK







### User Guide

Norsk



# Mirage™ SoftGel




Fitting / Maskinpassning / Tilpasning / Tilpasning /  
Sovitusohje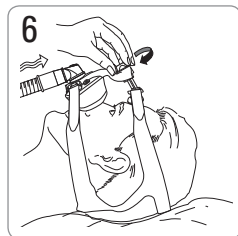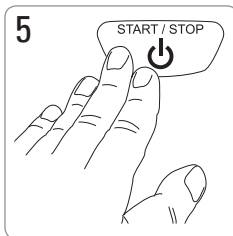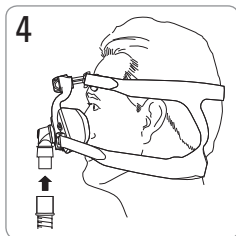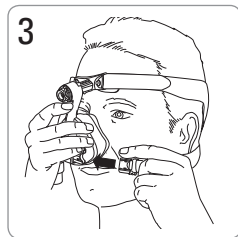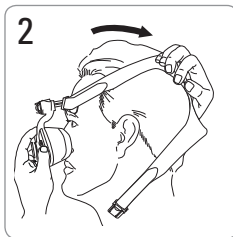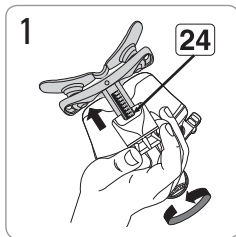

Removal / Avlägsnande / Aftagning / Fjerne masken /  
Irrottaminen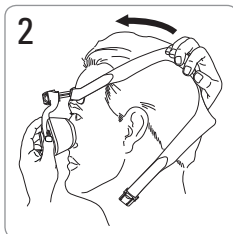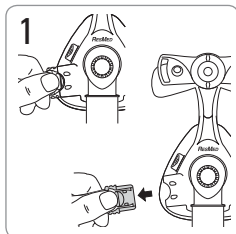

# Mirage SoftGel

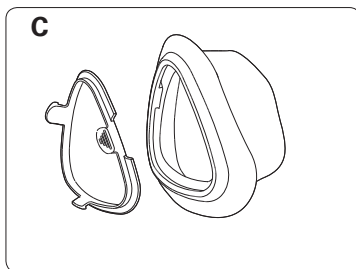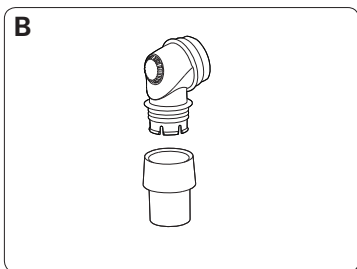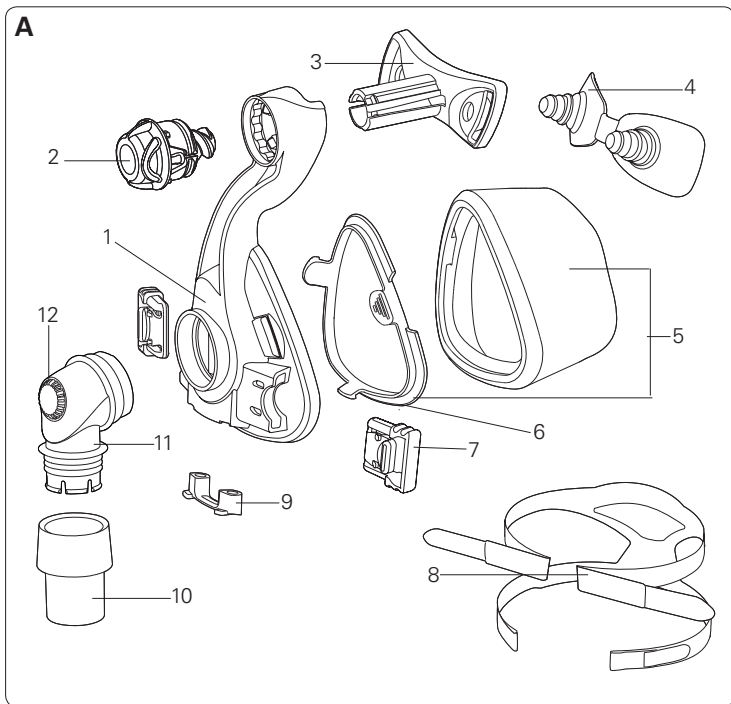**Mask components / Maskkomponenter /  
Maskekomponenter / Maskens komponenter /  
Maskin osat**

<b>Item / Produkt / Vare / Artikkel / Osa</b>	<b>Description / Beskrivning / Beskrivelse / Beskrivelse / Nimitys</b>	<b>Part number / Artikelnummer / Delnummer / Artikelnummer / Tuotenumero</b>
1	Mask frame / Maskram / Maskeramme / Maskeramme / Maskin runko	60180 (St) 60193 (S)
2	Dial / Kontrollratt / Reguleringsknapp / MicroFit-bryter / Säädin	61289
3	Forehead support / Pannstöd / Pandestøtte / Pannestøtte / Otsatuki	16393
4	Forehead support pad / Pannstödsdyna / Pandepude / Pute til pannestøtte / Otsatuen pehmike	60123 (1) 60124 (10)
5	Soft Gel Cushion & clip / Mjukdel & clips / Næsepude og låsering / Cushion og klips / Pehmike ja kiinnike	61631 (S) 61632 (M) 61633 (L) 61634 (LW)
6	Cushion clip / Mjukdelsclips / Låsering / Cushionklips / Pehmikkeen kiinnike	60176 (St) 60197 (S)
7	Headgear clip / Clips för huvudbandet / Hovedbåndscips / Klips til hodestropp / Pääremmien kiinnike	16569 (2) 16734 (10)
8	Headgear / Huvudband / Hovedbånd / Hodestropper / Pääremmit	16118 (S) 16117 (M) 16119 (L)
9	Ports cap / Portlock / Porthætte / Porthette / Liitäntöjen korkki	16570 (2) 16571 (10)
10	Swivel / Swivel / Drejestuds / Svivel / Pyörivä liitin	16565
11	Elbow / Knärör / Vinkelstykke / Kneledd / Kulmakappale	16387
12	Vent / Ventil / Lufthuller / Luftventil / Ilma-aukko	-

A	Complete system / Komplet system / Komplet maske / Komplet maskesystem / Maski täydellisenä	61636 (S) 61637 (M) 61638 (L) 61639 (LW)
B	Elbow assembly / Knärörsmontage / Vinkelsamling / Kneleddsett / Kulmakappalesarja	16399
Also available / Övrigt / Desuden fås / Leveres også / Saatavissa myös		
C	Activa LT Cushion & clip / Mjukdel & clips / Næsepude og låsering / Cushion og klips / Pehmike ja kiinnike	60198 (S) 60177 (M) 60178 (L) 60179 (LW)

S Small / Small / Small / Liten / Pieni










M Medium / Medium / Medium / Medium / Keskikokoinen

L Large / Large / Large / Stor / Suuri













LW Large Wide / Large Wide / Large Wide / Stor Vid / Suuri, leveä

St Standard / Standard / Standard / Standard / Vakio

# Disassembly / Demontering / Demontering / Demontering / Purkaminen
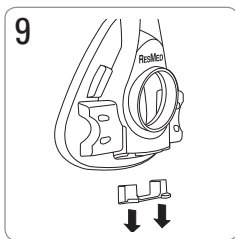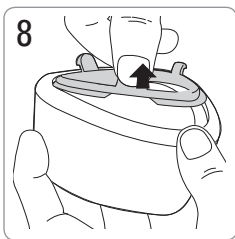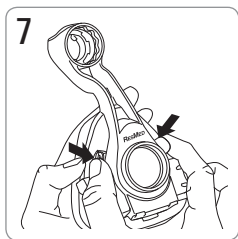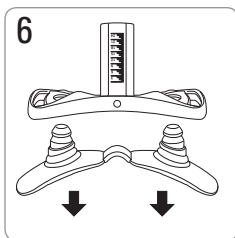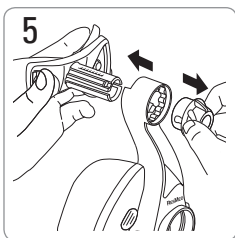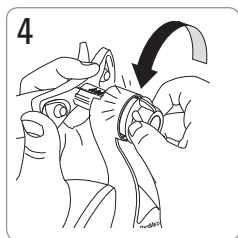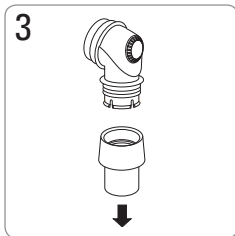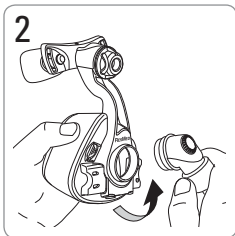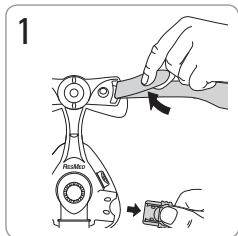
# Reassembly / Montering / Genmontering / Montering / Kokoaminen
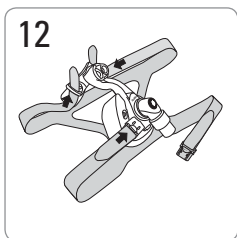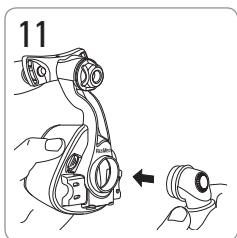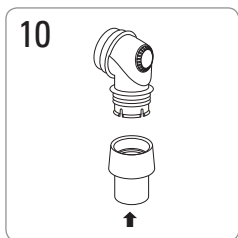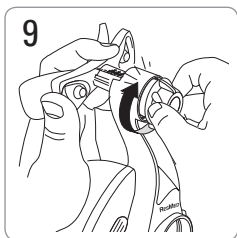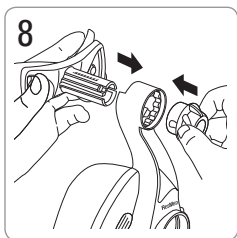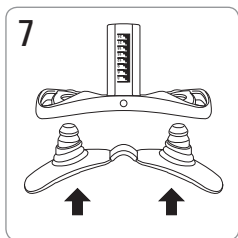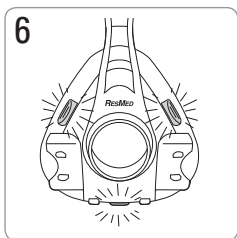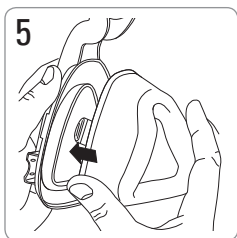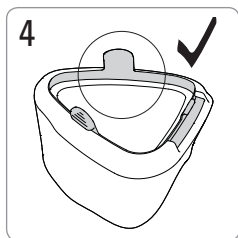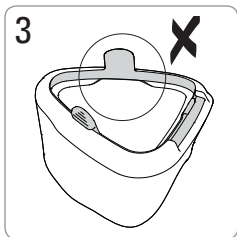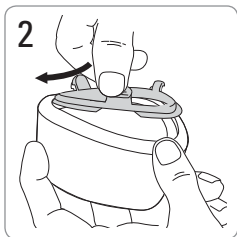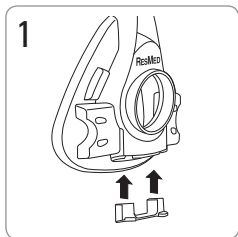
# Mirage™ SoftGel

## NESEMASKE

Takk for at du valgte Mirage SoftGel.

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### Innstillings- alternativer for masken

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
English • Svenska • Dansk • Norsk • Suomi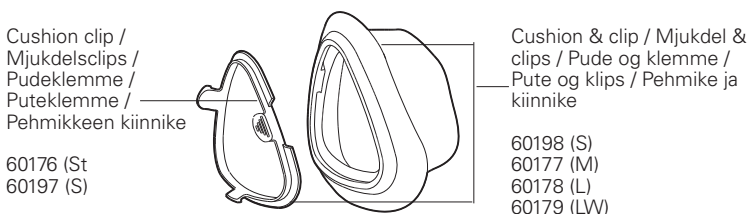

#### English

Please read this information in conjunction with your Mirage™ SoftGel User Guide when changing your Mirage SoftGel cushion to the Mirage Activa LT cushion.

#### Intended use

The Mirage Activa LT channels airflow noninvasively to a patient from a positive airway pressure device such as a continuous positive airway pressure (CPAP) or bilevel system.

The Mirage Activa LT is:

- to be used by adult patients (> 30 kg) for whom positive airway pressure has been prescribed
- intended for single-patient re-use in the home environment and multipatient re-use in the hospital/institutional environment.

## Troubleshooting

Problem/possible cause	Solution
------------------------	----------

### Mask is uncomfortable

Cushion incorrectly inflated	With the air turned off, adjust the straps so that you can insert two fingers between your cheek and the lower headgear strap.
------------------------------	--

## Technical specifications

<b>Dead space information</b>	The dead space of the mask varies according to cushion size. It is 185 mL for the Medium size.
<b>Resistance</b>	Drop in pressure measured (nominal) at 50 L/min: 0.1 cm H <sub>2</sub> O at 100 L/min: 0.7 cm H <sub>2</sub> O
<b>Sound</b>	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 30 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 22 dBA, with uncertainty 3 dBA.
<b>Gross dimensions</b>	Mask fully assembled – no headgear. S: 141 mm (H) x 91 mm (W) x 107 mm (D) M, L, LW: 154 mm (H) x 95 mm (W) x 117 mm (D)
<b>Mask setting options</b>	Select 'Activa' (if available) or 'Standard' as the mask option when using the Mirage Activa LT with ResMed devices that have mask setting options.

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© 2011 ResMed Ltd.
Mirage and Activa are trademarks of ResMed Ltd and are registered in U.S. Patent and Trademark Office.
ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA
ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK