

# Produktkompatibilitetslista för masker

Detta dokument visar de masker som är kompatibla med ResMeds övertrycksprodukter och eventuellt nödvändig maskinställning. För mer utförlig information om korrekt användning av dessa produkter hänvisas till respektive bruksanvisningar.

Obs! Vissa produkter finns inte i alla delar av världen.

## Hela ansiktsmasker

Produkt	AcuCare F1-4	AirFit™ F30	AirTouch™ F20 / AirTouch™ F20 for Her AirFit™ F20 / AirFit™ F20 for Her	AirFit™ F10 / AirFit™ F10 for Her	Mirage™	Mirage Series 2	Ultra Mirage™	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Hospital Full Face
AirMini	X	✓	✓	X	X	X	X	X	X	X	X
Air 10 Serie	N/V	Full Face	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
Lumis™	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
S9 AutoSet™ / S9 Elite™	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
S9 Escape™	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AutoSet CS™ 2	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE¹	FULL FACE
AutoSet CS™ (S9)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
AutoSet CS™ (S9 PaceWave) / AutoSet CS-A™ (S9 PaceWave)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
C-Series Tango™	N/V	N/V	N/V				Inställning ej nödvändig. Rådgör med klinikern innan du byter typ av mask.				
S7™ Lightweight	N/V	N/V	N/V				Inställning ej nödvändig. Rådgör med klinikern innan du byter typ av mask.				
S7 Elite™ / S7 AutoSet Spirit	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S7 AutoSet Respond™	N/V	N/V	N/V	MIRAGE	STANDARD	ULTRA	MIRAGE	MIRAGE	MIRAGE	MIR FULL	MIR FULL
S8™ Auto 25	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Spirit™	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Vantage™	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™	N/V	N/V	N/V				Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med klinikern och läs bruksanvisningen.				
S8 Escape™ II	N/V	N/V					Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med klinikern och läs bruksanvisningen.				
S8 Elite™ / S8 Elite™ II	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 AutoSet Spirit™ II	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™	N/V	N/V	N/V				Inställning ej nödvändig. Rådgör med klinikern innan du byter typ av mask.				
VPAP™ III / VPAP™ III ST	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	N/V	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ IV / VPAP IV ST	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Adapt SV	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE¹	FULL FACE
VPAP™ Adapt (S9)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face

Produkt	AcuCare F1-4	AirFit™ F30	AirTouch™ F20 / AirTouch™ F20 for Her AirFit™ F20 / AirFit™ F20 for Her	AirFit™ F10 / AirFit™ F10 for Her	Mirage™	Mirage Series 2	Ultra Mirage™	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Hospital Full Face
VPAP™ Adapt (S9 PaceWave)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Malibu	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ Auto 25	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
Auto 25 (S9)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ S / VPAP™ ST	N/V	N/V	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9) / VPAP™ ST-A (S9)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	N/V	N/V	Full Face	Full Face	N/V	N/V	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Tx	N/V	N/V	N/V	FULL FACE	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
Stellar™	N/V	N/V	FULL FACE	FULL FACE	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	✓	N/V	N/V	✓	N/V	N/V	✓	✓	N/V	✓	✓
VS III™	N/V	N/V	N/V	✓	N/V	N/V	✓	✓	N/V	✓	✓
Astral™	FULL FACE	N/V	FULL FACE	N/V	N/V	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE

<sup>1</sup> SmartStop fungerar eventuellt inte på det sätt som beskrivs om man använder masken med ResMed AutoSet CS 2 eller VPAP Adapt SV.

N/V = Masken ej validerad för användning med produkten.

✓ = Kompatibel X = Inte kompatibel

## Näsmasker 1

Produkt	AirFit™ N30i	AirFit™ N20 Classic	AirFit™ N20 / AirFit™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage FX™ for Her	Mirage Activa™ LT	Mirage™ SoftGel	Hospital Nasal	
AirMini™	X	✓	✓	X	X	X	X	X	X	
AirStart™ 10 / AirSense™ 10	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
AirCurve™ 10 CS Pacewave	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
AirCurve™ 10 CS-A Pacewave	X	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
Lumis™	X	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
S9 AutoSet™ / S9 Elite™	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
S9 Escape™	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
AutoSet CS™ 2	X	N/V	N/V	N/V	N/V	VISTA <sup>1</sup>	ACTIVA	ACTIVA	VISTA	
AutoSet CS™ (S9)	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
AutoSet CS™ (S9 PaceWave)	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
AutoSet CS-A™ (S9 PaceWave)	X	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal	
C-Series Tango™	N/V	N/V	N/V	N/V	N/V	Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.				
S7™ Lightweight	N/V	N/V	N/V	N/V	N/V	Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.				
S7 Elite™ / S7 AutoSet Spirit	N/V	N/V	N/V	N/V	N/V	STANDARD	STANDARD	STANDARD	STANDARD	
S7 AutoSet Respond™	N/V	N/V	N/V	N/V	N/V	STANDARD	STANDARD	STANDARD	STANDARD	
S8™ Auto 25	X	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	
S8 AutoSet Spirit™	N/V	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	
S8 AutoSet Vantage™	N/V	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD	

Produkt	AirFit™ N30i	AirFit™ N20 Classic	AirFit™ N20 / AirFit™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage FX™ for Her	Mirage Activa™ LT	Mirage™ SoftGel	Hospital Nasal
S8 Escape™	N/V	N/V	N/V		Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med kliniker och läs bruksanvisningen.				
S8 Escape™ II	N/V	N/V			Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med kliniker och läs bruksanvisningen.				
S8 Elite™ / S8 Elite™ II	N/V	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 AutoSet Spirit™ II	N/V	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
S8 Compact™	N/V	N/V	N/V		Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.				
VPAP™ III / VPAP™ III ST	X	N/V	N/V	MIRAGE	MIRAGE	STANDARD	STANDARD	STANDARD	N/V
VPAP™ III ST-A	X	N/V	N/V	MIRAGE	MIRAGE	STANDARD	STANDARD	STANDARD	N/V
VPAP™ IV / VPAP™ IV ST	X	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
VPAP™ Adapt SV	X	N/V	N/V	N/V	N/V	VISTA <sup>1</sup>	ACTIVA	ACTIVA	VISTA
VPAP™ Adapt (S9)	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal
VPAP™ Adapt (S9 PaceWave)	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal
VPAP™ Malibu	X	N/V	N/V	SWIFT	SWIFT	VISTA <sup>1</sup>	N/V	N/V	MERIDIAN
VPAP™ Auto 25	X	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
Auto 25 (S9)	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal
VPAP™ S / VPAP™ ST	X	N/V	SWIFT	SWIFT	SWIFT	STANDARD	ACTIVA	ACTIVA	STANDARD
VPAP™ S (S9) / VPAP™ ST (S9)	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal
VPAP™ ST-A (S9)	X	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	X	Kuddar	Kuddar	Kuddar	Kuddar	Nasal	Nasal	Nasal	Nasal
VPAP™ Tx	X	N/V	N/V	KUDDAR	KUDDAR	NASAL	NASAL	NASAL	NASAL
Stellar™	X	KUDDAR	KUDDAR	KUDDAR <sup>1</sup>	KUDDAR <sup>1</sup>	NASAL	NASAL	NASAL	NASAL
VS Ultra™ / VS Integra™	X	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	X	N/V	N/V	N/V	N/V	N/V	✓	N/V	✓
Astral™	X	Kuddar	Kuddar	N/V	N/V	Nasal	N/V	Nasal	Nasal

<sup>1</sup> SmartStop fungerar eventuellt inte på det sätt som beskrivs om man använder masken med ResMed AutoSet CS 2 eller VPAP Adapt SV, Stellar.

N/V = Masken ej validerad för användning med produkten.

✓ = Kompatibel X = Inte kompatibel

## Näsmasker 2

Produkt	Pixi™	Mirage Kidsta™	Mirage Micro™	Mirage Vista™	Ultra Mirage II	Mirage	Mirage Activa™	Modular	Papillon™	Meridian
AirMini™	X	X	X	X	X	X	X	X	X	X
Air 10 Serie	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
Lumis™	N/V	Nasal	Nasal	Nasal	Nasal	N/V	Nasal	N/V	N/V	Nasal
S9 AutoSet™ / S9 Elite™	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
S9 Escape™	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
AutoSet CS™ 2	N/V	N/V	VISTA <sup>1</sup>	VISTA	ULTRA	N/V	ACTIVA	N/V	PAPILLON	VISTA
AutoSet CS™ (S9)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
AutoSet CS™ (S9 PaceWave) / AutoSet CS-A™ (S9 PaceWave)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal
C-Series Tango™	N/V	N/V	N/V		Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.					N/V

Produkt	Pixi™	Mirage Kidsta™	Mirage Micro™	Mirage Vista™	Ultra Mirage II	Mirage	Mirage Activa™	Modular	Papillon™	Meridian	
S7™ Lightweight	N/V	N/V	N/V		Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.						N/V
S7 Elite™ / S7 AutoSet Spirit	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	STANDARD	N/V	
S7 AutoSet Respond™	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	STANDARD	N/V	
S8™ Auto 25	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD	
S8 AutoSet Spirit™	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD	
S8 AutoSet Vantage™	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD	
S8 Escape™	N/V	N/V			Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med kliniker och läs bruksanvisningen.						
S8 Escape™ II	N/V	N/V			Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med kliniker och läs bruksanvisningen.						
S8 Elite™ / S8 Elite™ II	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD	
S8 AutoSet Spirit™ II	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	STANDARD	STANDARD	STANDARD	
S8 Compact™	N/V	N/V			Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.						
VPAP™ III / VPAP™ III ST	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	N/V	N/V	
VPAP™ III ST-A	N/V	STANDARD	STANDARD	STANDARD	ULTRA	MIRAGE	STANDARD	STANDARD	N/V	N/V	
VPAP™ IV / VPAP™ IV ST	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD	
VPAP™ Adapt SV	N/V	N/V	VISTA <sup>1</sup>	VISTA	ULTRA	N/V	ACTIVA	N/V	N/V	VISTA	
VPAP™ Adapt (S9)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal	
VPAP™ Adapt (S9 PaceWave)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal	
VPAP™ Malibu	N/V	N/V	MERIDIAN	VISTA	ULTRA	N/V	N/V	N/V	N/V	MERIDIAN	
VPAP™ Auto 25	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD	
Auto 25 (S9)	N/V	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	STANDARD	
VPAP™ S / VPAP™ ST	N/V	N/V	STANDARD	STANDARD	ULTRA	MIRAGE	ACTIVA	N/V	N/V	STANDARD	
VPAP™ S (S9) / VPAP™ ST (S9) / VPAP™ ST-A (S9)	Pediatrik	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal	
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	Pediatrik	N/V	Nasal	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	Nasal	
VPAP™ Tx	N/V	NASAL	NASAL	NASAL	NASAL-ULTRA	N/V	NASAL	N/V	N/V	NASAL	
Stellar™	Pediatrik	NASAL	NASAL	NASAL	ULTRA	N/V	NASAL	N/V	N/V	N/V	
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	
VS III™	N/V	✓	✓	✓	✓	N/V	✓	N/V	N/V	N/V	
Astral™	Pediatrik	Nasal	Nasal	Nasal	Nasal Ultra	N/V	N/V	N/V	N/V	N/V	

N/V = Masken ej validerad för användning med produkten.

✓ = Kompatibel X = Inte kompatibel

## Näskuddar

Produkt	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Mirage Swift™	Mirage Swift™ II	Swift™ FX / Swift™ FX for Her / Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT	Swift™ LT for Her / Swift™ LT-F
AirMini™	✓	X	X	X	X	X	X
Air 10 Serie	X	Kuddar <sup>2</sup>	N/V	Kuddar	Kuddar	Kuddar	Kuddar
Lumis™	X	Kuddar <sup>2</sup>	N/V	Kuddar	Kuddar	Kuddar	Kuddar
S9 AutoSet™ / S9 Elite™	X	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
S9 Escape™	X	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar

Produkt	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Mirage Swift™	Mirage Swift™ II	Swift™ FX / Swift™ FX for Her / Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT	Swift™ LT for Her / Swift™ LT-F
AutoSet CS™ 2	X	N/V	N/V	N/V	N/V	N/V	N/V
AutoSet CS™ (S9)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
AutoSet CS™ (S9 PaceWave) / AutoSet CS-A™ (S9 PaceWave)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
C-Series Tango™	X				Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.		
S7™ Lightweight	X				Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.		
S7 Elite™ / S7 AutoSet Spirit	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S7 AutoSet Respond™	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8™ Auto 25	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 AutoSet Spirit™	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 AutoSet Vantage™	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	X				Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med kliniker och läs bruksanvisningen.		
S8 Escape™ II	X				Kräver tryckkalibrering vid 12 cm H <sub>2</sub> O. Rådgör med kliniker och läs bruksanvisningen.		
S8 Elite™ / S8 Elite™ II	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 AutoSet Spirit™ II	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	X				Inställning ej nödvändig. Rådgör med kliniker innan du byter typ av mask.		
VPAP™ III / VPAP™ III ST	X	MIRAGE <sup>2</sup>	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	X	MIRAGE <sup>2</sup>	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ IV / VPAP™ IV ST	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Adapt SV	X	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
VPAP™ Adapt (S9 PaceWave)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
VPAP™ Malibu	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Auto 25	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
Auto 25 (S9)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
VPAP™ S / VPAP™ ST	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S (S9) / VPAP™ ST (S9) / VPAP™ ST-A (S9)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
VPAP™ ST (S9 iVAPS) / VPAP™ ST-A (S9 iVAPS)	X	Kuddar <sup>2</sup>	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar
VPAP™ Tx	X	KUDDAR <sup>2</sup>	KUDDAR	KUDDAR	KUDDAR	KUDDAR	KUDDAR
Stellar™	X	KUDDAR <sup>1,2</sup>	KUDDAR <sup>1</sup>	KUDDAR <sup>1</sup>	KUDDAR <sup>1</sup>	KUDDAR <sup>1</sup>	KUDDAR <sup>1</sup>
VS Ultra™ / VS Integra™	X	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	X	N/V	N/V	N/V	N/V	N/V	N/V
Astral™	X	N/V	Kuddar	Kuddar	Kuddar	Kuddar	Kuddar

<sup>1</sup> SmartStop fungerar eventuellt inte på det sätt som beskrivs om man använder masken med Stellar.

<sup>2</sup> Precisionen för tidalvolym/minutventilation kan minska vid användning av AirFit P10-masken i bilevelfunktioner särskilt vid högre tryckstöd.

N/V = Masken ej validerad för användning med produkten.

✓ = Kompatibel X = Inte kompatibel



 **ResMed Ltd**  
1 Elizabeth Macarthur Drive  
Bella Vista NSW 2153 Australien

Information om andra ResMed kontor hittar du på [ResMed.com](https://www.resmed.com). För information om patent och övrig intellektuell egendom, se [ResMed.com/ip](https://www.resmed.com/ip). AcuCare, Lumis, AirFit, AirTouch, AirMini, S9, S9 AutoSet, S9 Elite, S9 Escape, AutoSet, Tango, S8, S8 AutoSet Vantage, S8 Escape, S8 Elite, S8 Compact, VPAP, VPAP Adapt SV, VPAP Tx, Stellar, Mirage, Mirage Micro, Mirage Vista, Mirage Kidsta, Ultra Mirage, Activa, Pixi, Swift, Mirage Swift och Quattro är varumärken tillhöriga ResMed Ltd och Astral, Tango, AutoSet, Escape, S9, VPAP, Adapt SV, Stellar, Mirage, Quattro, Activa, Kidsta, Swift och Vista är registrerade hos U.S. Patent and Trademark Office. Astral, VS Ultra, VS Integra och VS III är varumärken tillhöriga ResMed Paris SAS. © 2018 ResMed Ltd. 1017550/11 2018-11

[ResMed.com](https://www.resmed.com)

 0123