



ResMed

AirFit™ F30




Simplicity wrapped in high performance

Introducing ResMed AirFit™ F30 – our minimal-contact full face mask that outperforms Phillips Respironics® DreamWear™ full face and Amara View™ in key areas that matter to you – comfort,¹⁻³ fit,^{2,3} seal¹⁻³ and quietness.⁴⁻⁶ Plus, it makes ordering and inventory management more efficient for your business with the fewest SKUs.*

ResMed.com/AirFitF30



AirFit F30 highlights

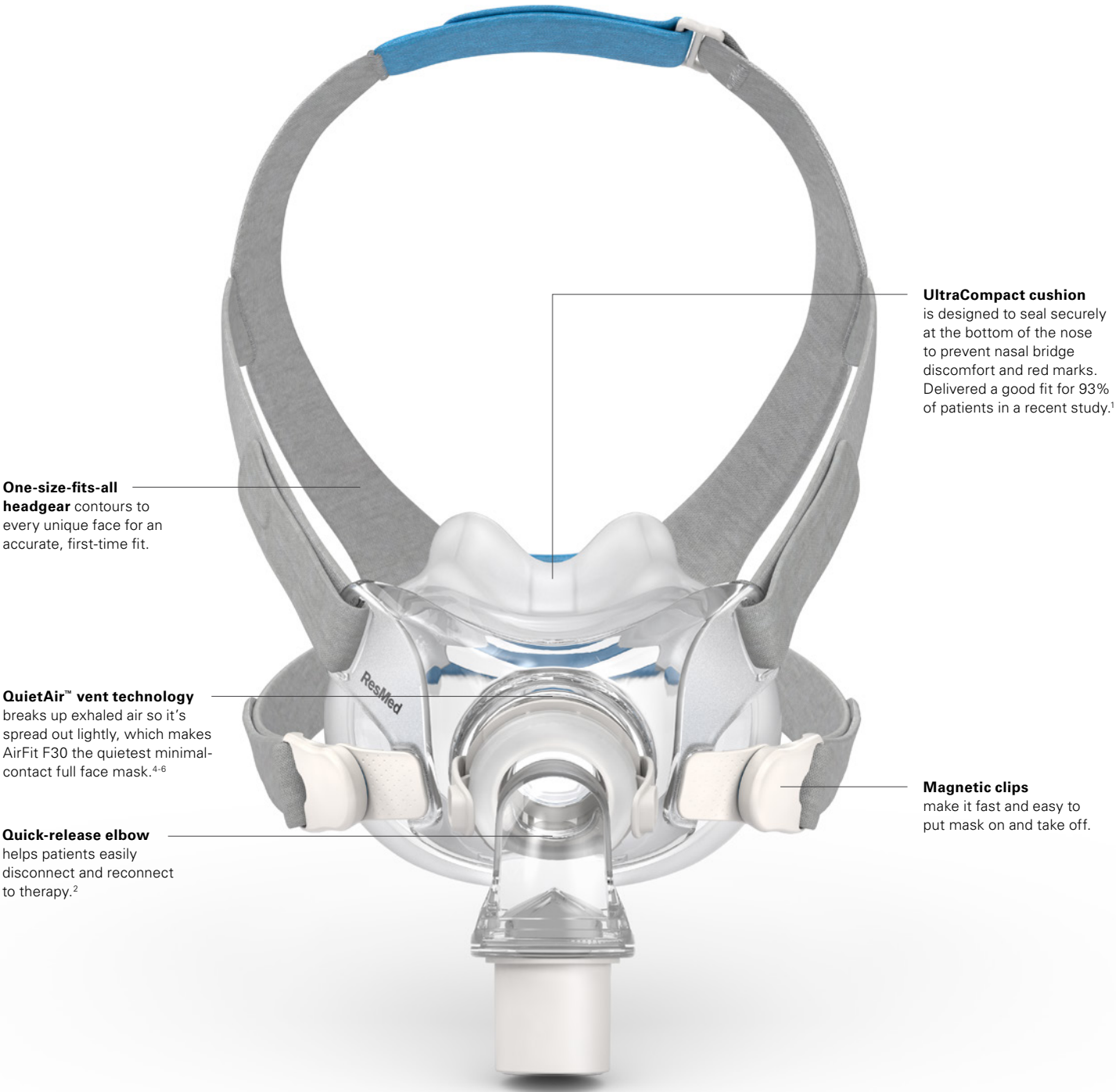

One-size-fits-all headgear contours to every unique face for an accurate, first-time fit.

UltraCompact cushion is designed to seal securely at the bottom of the nose to prevent nasal bridge discomfort and red marks. Delivered a good fit for 93% of patients in a recent study.¹

QuietAir™ vent technology breaks up exhaled air so it's spread out lightly, which makes AirFit F30 the quietest minimal-contact full face mask.⁴⁻⁶


Magnetic clips make it fast and easy to put mask on and take off.

Quick-release elbow helps patients easily disconnect and reconnect to therapy.²

AirFit F30 Product Codes

Complete mask systems (includes frame system + headgear)	Cushions included	Product code
AirFit F30: Small	S	64100
AirFit F30: Medium	M	64101
Frame systems (includes frame, cushion and elbow)		
AirFit F30 Frame System: Small	S	64155
AirFit F30 Frame System: Medium	M	64156

Accessories	Product code
AirFit F30 Cushion: Small	64150
AirFit F30 Cushion: Medium	64151
AirFit F30 Headgear: Standard	64161

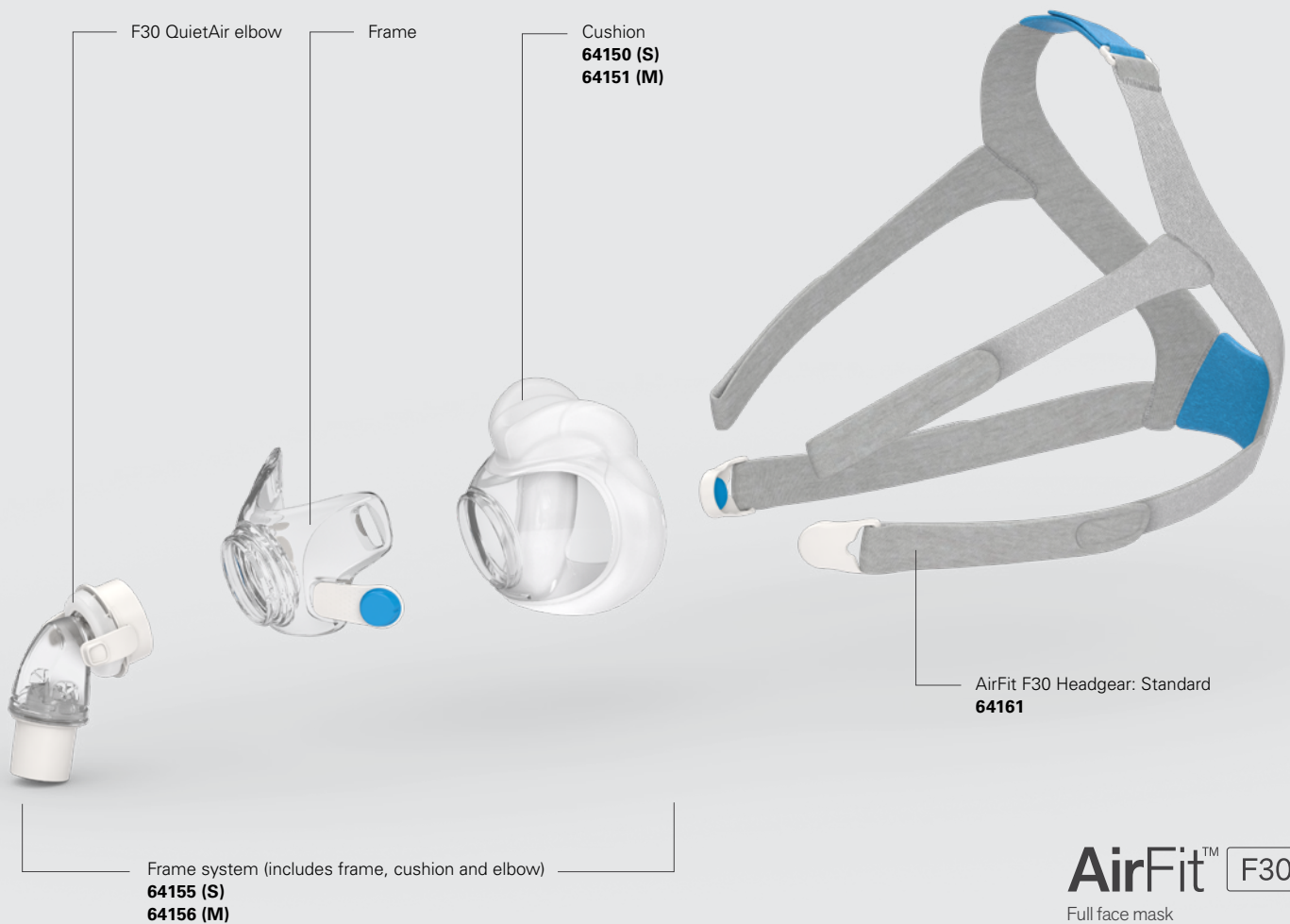

AirFit™ F30
Full face mask



AirFit™ F30
Full face mask



Fitting guide for AirFit F30



- Place the UltraCompact cushion under your nose and ensure it sits comfortably on your face. With the ResMed logo on the headgear facing up, pull the headgear over your head.



- Bring the lower headgear straps under your ears and attach the magnetic headgear clips to the frame.



- Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.



- If the headgear is too close to your ear, adjust the headgear strap on the crown of the head.



- Connect the air tubing from your CPAP machine to the elbow.



- Reattach the elbow to the frame. The mask is now ready to use.



Additional options

For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

*ResMed AirFit F30 had fewer SKUs than Philips Respironics DreamWear full face and Amara View at the time of this study.

1 ResMed AirFit F30 fitting study of 75 patients conducted from April to May 2018 in multiple US locations. Study evaluated AirFit F30 against Philips Respironics Amara View (both masks are minimal-contact, full face). **2** ResMed AirFit F30 guided clinical study of 23 patients conducted from June 8 to August 24 2018 in Australia. Study tested ResMed AirFit F30 against Philips Respironics DreamWear full face (both masks are minimal-contact, full face). **3** ResMed AirFit F30 guided clinical study of 21 patients conducted from April to May 2018 in Australia. Study tested ResMed AirFit F30 against Philips Respironics Amara View. **4** ResMed. ResMed AirFit F30 user guide. Bella Vista, Australia; 2018. **5** Koninklijke Philips N.V. Philips Respironics DreamWear full face mask instructions for use. Murrysville, PA; 2016. **6** Koninklijke Philips N.V. Philips Respironics Amara View minimal-contact full face mask instructions for use. Murrysville, PA; 2016.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit and QuietAir are trademarks and/or registered trademarks of the ResMed family of companies. Amara View, DreamWear and Philips Respironics are trademarks of Koninklijke Philips N.V., registered in the US and other countries. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2018 ResMed. 10110769/2 2018-09

ResMed.com/AirFitF30