


Mirage Micro Nasal Mask Fitting Template

How to use the fitting template:

1. Cut out template
2. Cut out nasal holes
(hint: folding each nasal hole in half makes it easier to cut)
3. Follow directions on template to determine appropriate cushion size

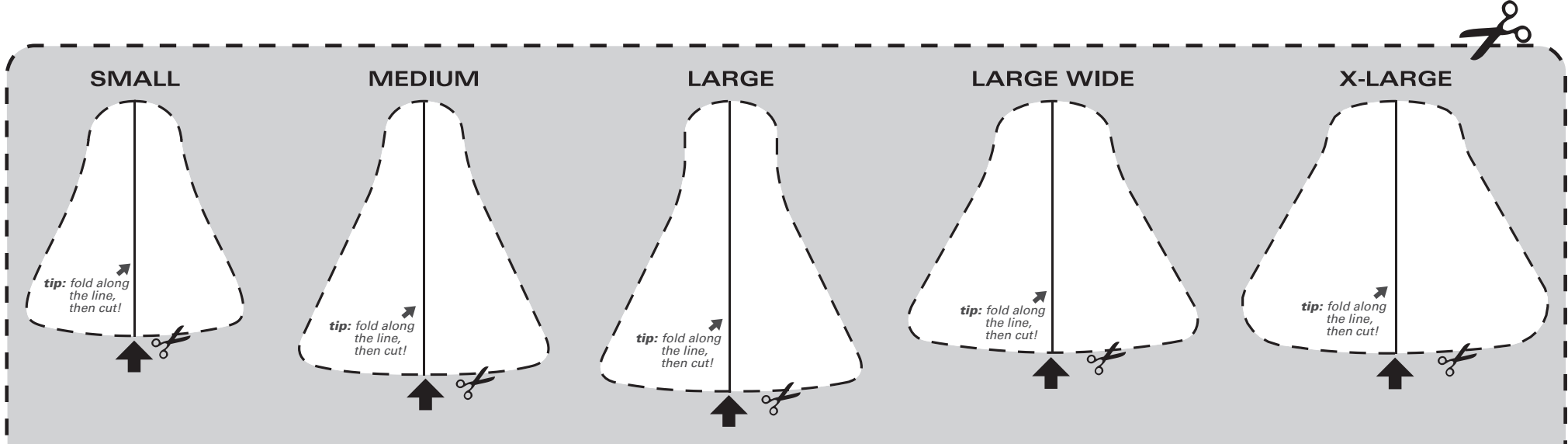


RESMED

Mirage Micro™

NASAL MASK


Fitting Template

Note: Mask fitting templates should be used to aide patients in the fitting of CPAP masks. Altering the size or dimensions of the original online PDF in any way may compromise the quality and accuracy of this template. The use of this template is not intended to replace the advice of a medical professional, sleep technician or HME dealer.

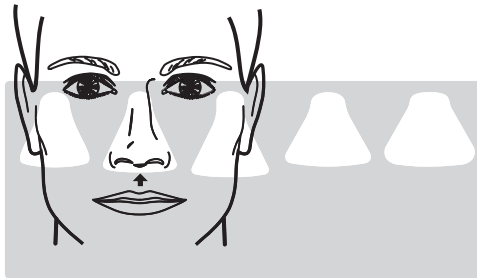


1

Place template around the nose and align arrow with bottom of nose

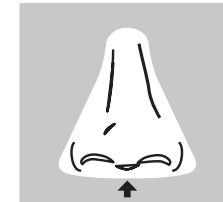

2

Obtain a close fit around the nose allowing some clearance around the nostrils and nasal bridge

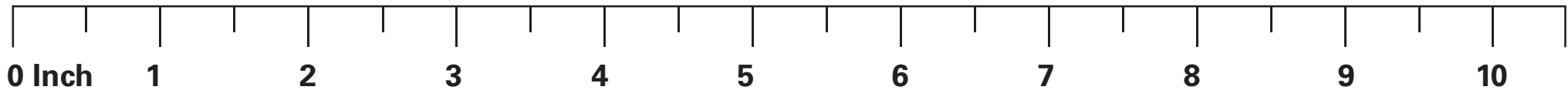

Actual Size: 10⁵/₁₂ inch (Width) X 5 inch (Height)

Ruler is for scaling purposes. To ensure this print is accurate, match a ruler with the ruler above.