

ResMed

Quattro™ Air NV
Non-vented full face mask



Delivering NIV... the lighter way

A busy healthcare environment requires a mask that delivers high-quality treatment in an efficient yet patient-friendly design. Likewise, a patient on long-term noninvasive ventilation (NIV) requires a mask that's comfortable and easy to use. Our Quattro™ Air NV full face mask does it all in a lightweight mask designed to meet a range of needs in the hospital and the home.

Comfort

The SoftEdge™ headgear is made of premium rolled-edge fabric that's gentle on the skin. Its lightweight design can also help minimize skin breakdown to enhance patient comfort.

Efficiency

The Spring Air™ cushion is designed to provide a secure seal, making NIV easy and effective.


Simplicity

With only five parts, the mask can be assembled in minimal time. And thanks to its quick-release clip with cord, it can be removed quickly and easily in an emergency.


Taking care of business

The Quattro Air NV delivers effective noninvasive ventilation to your patients while saving you time and money. It has just five parts, so it's quick and easy to assemble, disassemble, clean and reprocess for safe multi-patient use. With the Quattro Air NV taking care of your business needs, you can spend more time taking care of what's most important — your patients.

Quattro Air NV features and benefits

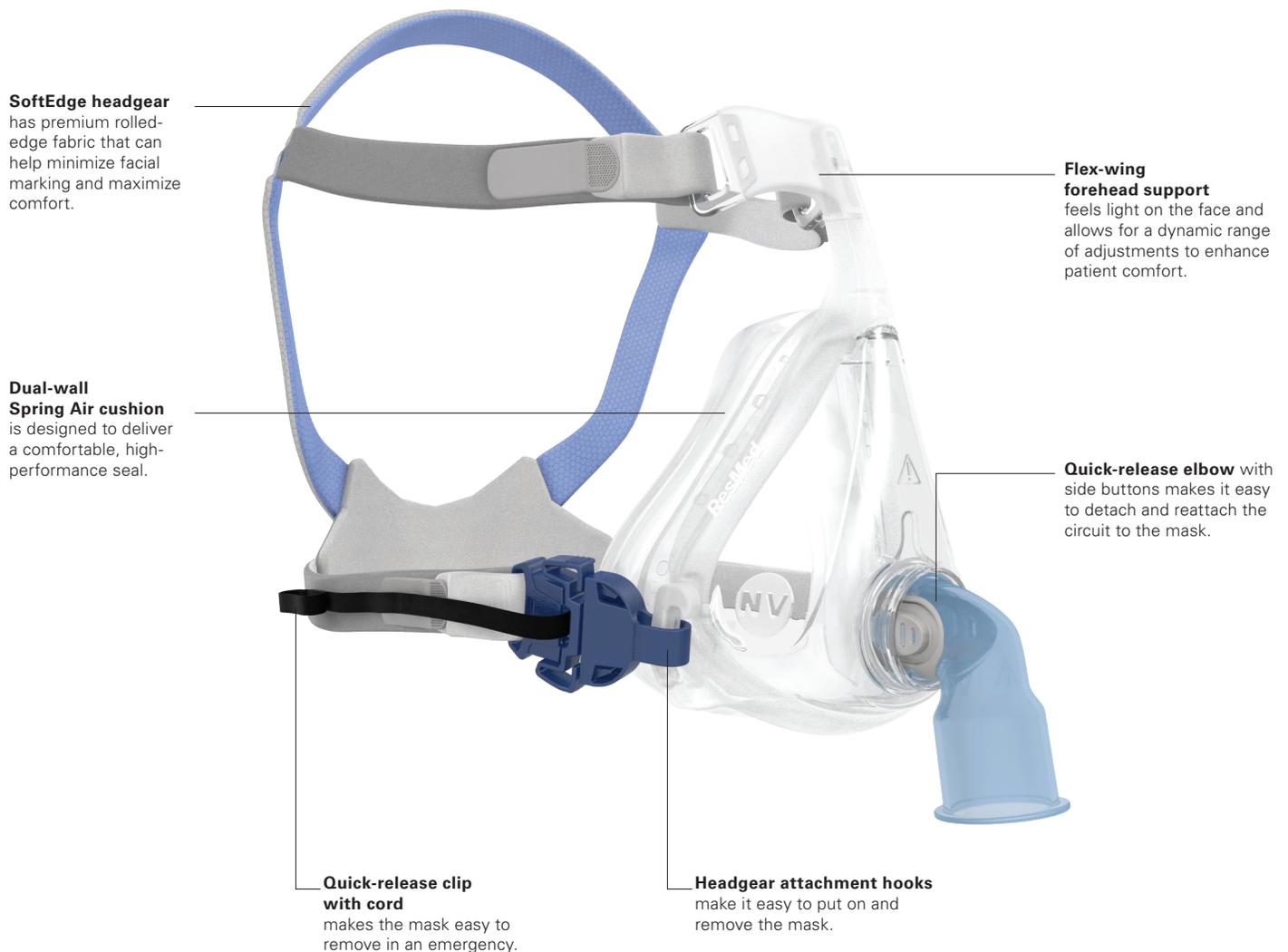

Quattro Air NV components

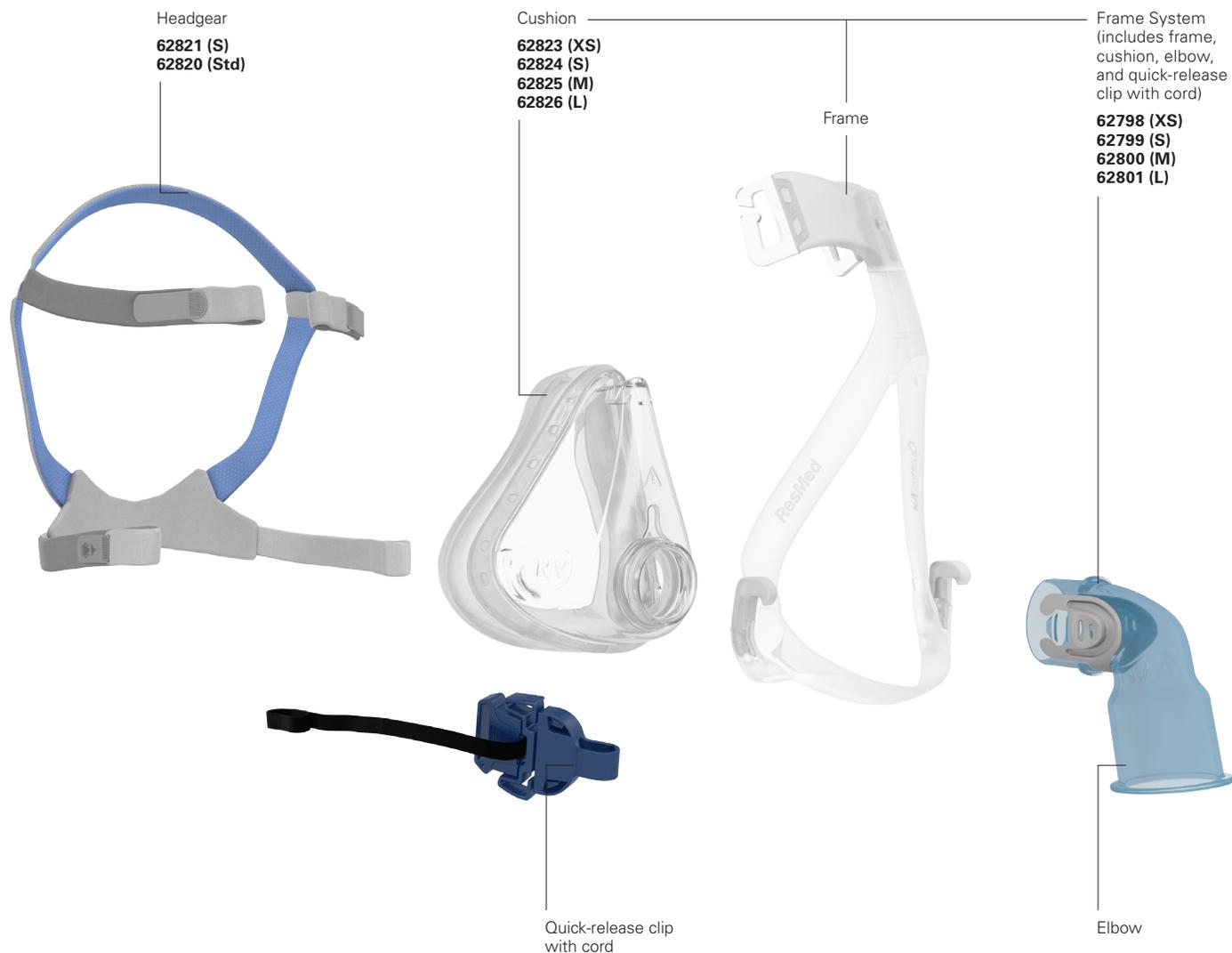

The XS mask system comes with the Small-sized headgear.
 The S, M and L mask systems come with the Standard-sized headgear.

Note: For more details, refer to the Quattro Air NV user guide.

Product codes

(Complete mask system)


	USA
Quattro Air NV mask system XS	62778
Quattro Air NV mask system S	62779
Quattro Air NV mask system M	62780
Quattro Air NV mask system L	62781

Quattro™ Air NV

Non-vented full face mask

Quattro Air NV sizing tips

Use the fitting template below as a guide to size your patient for the Quattro Air NV. Ensure that the patient's mouth is closed and relaxed when sizing.

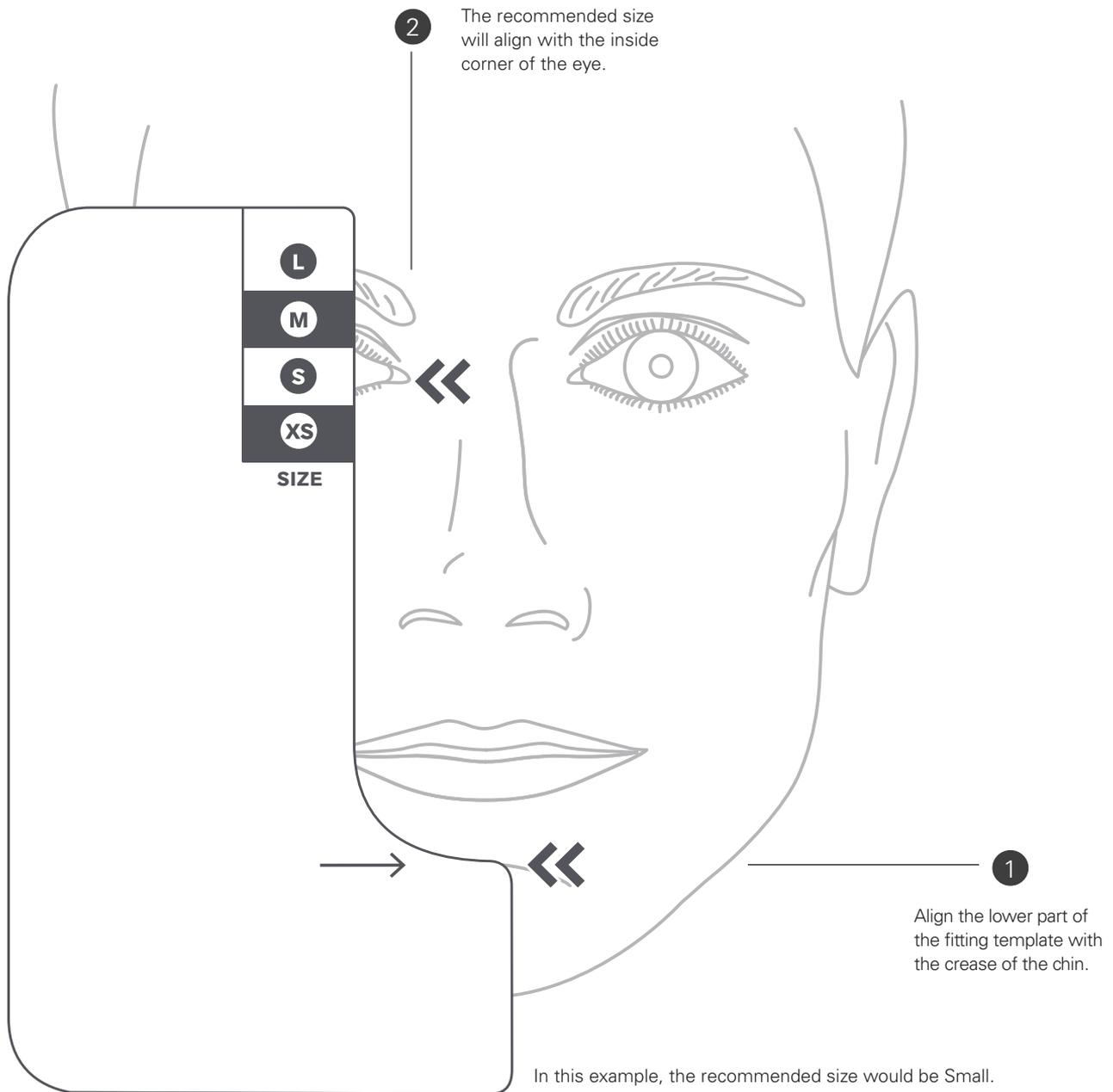

Fitting guide for the Quattro Air NV



- Remove the elbow from the mask by pressing the side buttons and pulling the elbow away. Then unclip the quick-release clip (not using the cord) to release the lower headgear straps.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head. Ensure that the label on the back of the headgear faces outwards when the headgear is put on.



- Undo the fastening tabs on the upper headgear straps. Pull the straps evenly until the mask is stable and positioned as shown above. Reattach the fastening tabs.



- Bring the lower straps below the ears. Loop the headgear onto the frame and snap in the quick-release clips together.



- Undo the fastening tabs on the lower headgear straps. Pull the straps evenly until the mask sits firmly on the chin and reattach the fastening tabs.



- Start ventilation and then connect the air tubing from the device to the elbow. Attach the elbow to the mask by pressing the side buttons and pushing the elbow into the mask.



- Readjust the forehead straps so that the mask fits comfortably and creates an optimal seal. Do not overtighten, as it may cause leaks.




- The mask and headgear should be positioned as shown above.



Removing your mask using the quick-release cord:

- Pull the black clip with cord down; the quick-release clip will separate from the mask.
- Pull the mask away from the face and back over the head.

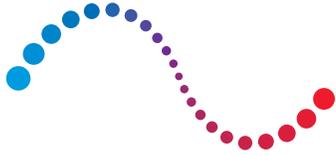

ResMed

Quattro™ Air NV
Non-vented full face mask

Optimal cushion positioning



Align the bottom of the cushion with the crease of the chin, and the top of the cushion with the corner of the eye.



Quattro Air NV sits on the nose at or below eye level.